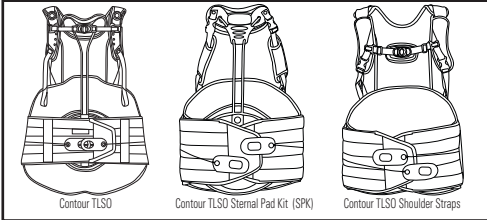
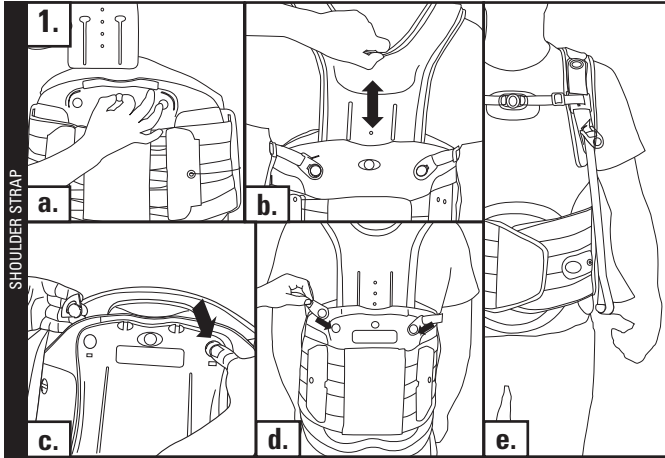
CONTOUR™ TLSO AND STERNAL PAD KIT OPTION

Aspen Medical Products, Inc.
 6481 Oak Canyon, Irvine, CA 92618
 800-295-2776 • 949-681-0200
www.aspenmp.com

ENGLISH
TLSO STRAP INSTALLATION

For Contour LSO instructions, please log onto www.aspenmp.com
 a. Pull the yellow rivet to release the extension panel. Pull the panel out.
 b. Slide the shoulder straps into the Contour panel. Lock the yellow rivet into place on the correct position. Adjust to the proper height setting and then lock into place with the yellow rivet.
 c. Remove the plastic fillers from the holes at the top of the Contour back panel.
 d. Pull and tighten the straps
 Note: If straps dig into underarms, lower the height setting on the back panel.

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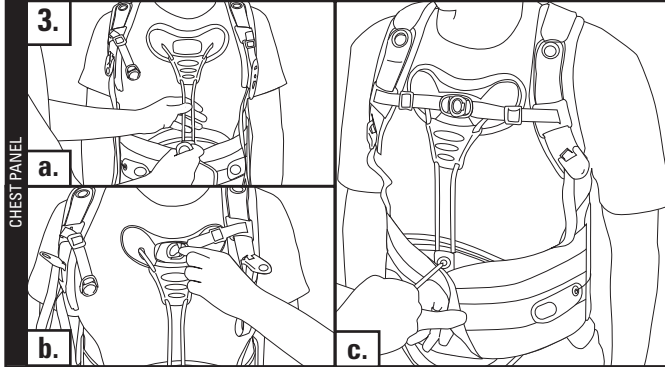
ENGLISH
CHEST PANEL ADJUSTMENT

a. Find the correct height and tighten the screws to lock the chest panel in place.
 b. Slip the buckles from the shoulder straps through the chest panel and snap into place.
 c. If needed, loosen the screws and adjust the bar height, then re-tighten the screw.

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ENGLISH

CONTOUR™ TLSO SPK

The Contour™ TLSO with Sternal Pad Kit Option is used to support and stabilize the thoracic lumbar-sacral portion of the spine for both non-operative and post-operative indications.

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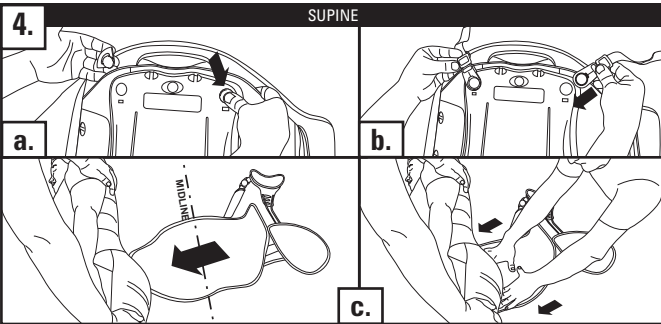
ENGLISH

CONTOUR TLSO SPK PREPARATION

- Take the Contour back panel and remove the plastic fillers from the holes on top.
- Snap the SPK kit straps into those holes.
- Gently logroll the patient and position the back panel so the patient will roll back onto it.

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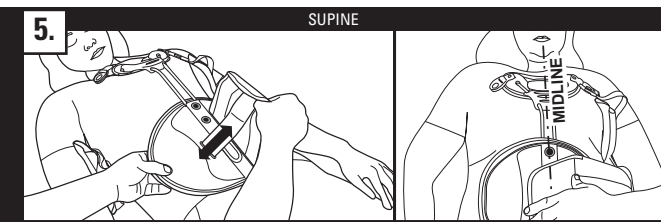
ENGLISH

SPK PLACEMENT

Center the RAP and chest panel. Slide the RAP on the adjustment strap as needed to fit the midline.

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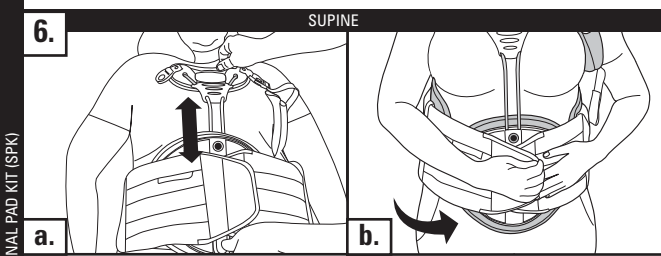
ENGLISH

SPK ADJUSTMENT

- Slide the chest panel to the desired height and tighten the thumbscrews to secure in place.
- Overlap the tapered ends of the QuikDraw and mate the Velcro snugly.

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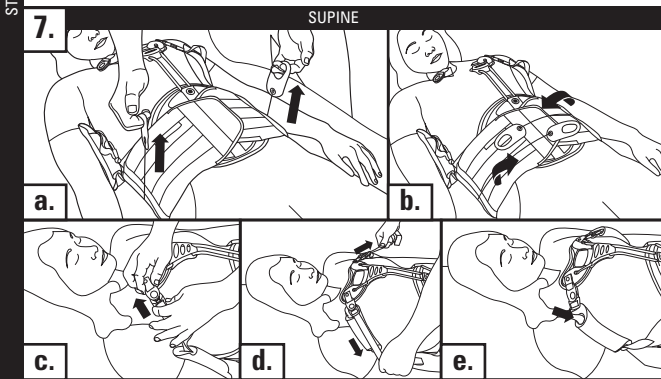
ENGLISH

SPK TIGHTENING

- Peel the pull-tabs away from the brace, and pull out and away to tighten.
- When tightened, place the pull-tabs down onto the Velcro and press down to secure.
- Clip the SPK strap into the chest panel.
- Pull down on the SPK straps to tighten.
- Extra strap can be tucked into the pad on the shoulder strap.

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LIMITED WARRANTY
Aspen Medical Products, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products in the event of breach of warranty shall be to repair or replace the defective product or part(s).
Aspen Medical Products shall have no obligation under this limited warranty in the event:
(a) The product was not purchased from Aspen Medical Products or through its authorized channels of distribution;
(b) The product is altered;
(c) Any parts not supplied by Aspen Medical Products are inserted into the product; or
(d) The product is not used in accordance with the Aspen Medical Products, Instructions for Use.

THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF, OR DEFECT IN, THE PRODUCT. ASPEN MEDICAL PRODUCTS SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS.
This warranty gives you specific legal rights and you may have other rights which vary from state to state.
Warning: To ensure proper use, please review all material.
A listing of authorized distributors in your area is available upon request.

READ INSTRUCTIONS BEFORE USE. Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations.

THIS PRODUCT IS INTENDED FOR APPLICATION BY LICENSED HEALTH CARE PRACTITIONERS AS DIRECTED BY A PHYSICIAN OR OTHER QUALIFIED MEDICAL AUTHORITY. PERMANENT INJURY MAY RESULT FROM USE OTHER THAN AS DIRECTED BY A PHYSICIAN OR OTHER MEDICAL AUTHORITY. ASPEN MEDICAL PRODUCTS SPECIFICALLY DISCLAIMS ANY OBLIGATION OR LIABILITY FOR INJURY OR DAMAGE BY REASON OF ANY APPLICATION OF THIS PRODUCT. PLEASE SEE THE ACCOMPANYING LIMITED WARRANTY FOR FURTHER INFORMATION IN THIS REGARD.

PATENT PENDING