Vista
LUMBAR ORTHOSIS

EXTRA SMALL
26 - 31 inches
66 - 78 cm

SMALL
30 - 37 inches
76 - 94 cm

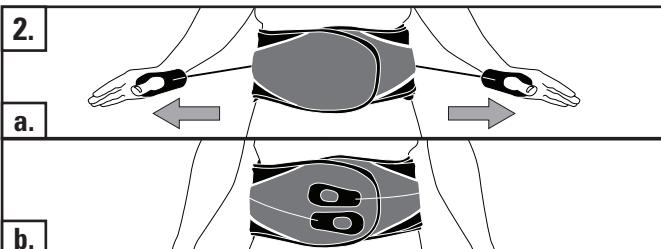
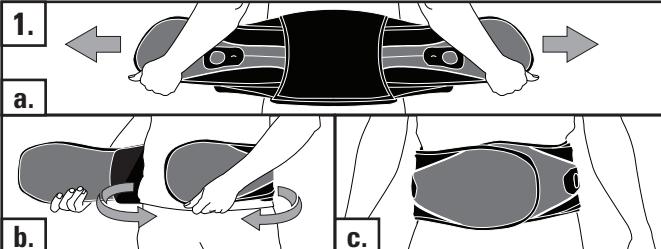
MEDIUM
36 - 43 inches
91 - 109 cm

LARGE
42 - 49 inches
107 - 124 cm

EXTRA LARGE
48 - 55 inches
122 - 140 cm

EXTRA EXTRA LARGE
54 - 60 inches
137 - 152 cm

Vista 627 LUMBAR ORTHOSIS Vista 631 LUMBOSACRAL ORTHOSIS Vista 637 LUMBOSACRAL ORTHOSIS

ENGLISH

ESPAÑOL

DEUTSCH

ITALIANO

FRANÇAIS

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BELT SIZING

- Fit male rivets of belt through proper sizing adjustment holes in sides of back panel.
- For smaller sizes, slide side panel through sizing adjustment slot and then fit rivets through proper adjustment holes from the under side.
- Once proper size is achieved, pull taught to lock rivets in place.

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* The Vista® 627 sizing follows closely to standard pants sizing. Size ranges are only guidelines. Your experiences and personal assessment could make the difference in choosing the correct size. For the Vista® 631, 637 and 464 add 2 inches for a better fit.

ENGLISH

ESPAÑOL

DEUTSCH

ITALIANO

FRANÇAIS

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PLACEMENT

- Center back panel in the middle of the back as low as is comfortable.
 - Wrap left side of belt tightly to the front of the body.
 - Repeat with right side, and while inhaling, press the hook loop material to achieve a snug fit.
- Note: Be sure top and bottom of brace is snug. If needed, angle ends to create a snug fit.

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FRANÇAIS

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TIGHTENING

- Pull strongly away from body. Tighten more than you need and then loosen to achieve a proper fit. Pull left tab to tighten lower, pull right to tighten upper as needed.
- Mate the pull tabs to loop material to maintain compression.

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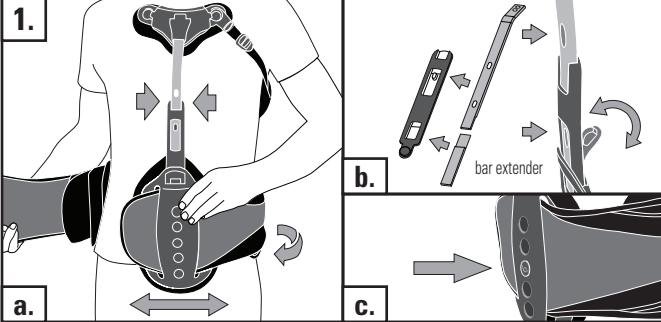
Aspen Medical Products, Inc.
6481 Oak Canyon, Irvine, CA 92618
800-295-2776 • 949-681-0200
www.aspenmp.com

EC

REP

ENGLISH

PLACEMENT & ADJUST

- Center back panel as low as possible. Slide abdominal panel and sternal pad sideways as needed to ensure brace and panel are at patient's midline.
- For shorter patients, remove bar extender from the abdominal panel sheath. Pry button back and slide aluminum strut so that the button locks into the appropriate hole.
- To further adjust the sternal pad on the abdominal panel, press the button with leaf logo inward and raise or lower the sternal pad as needed.

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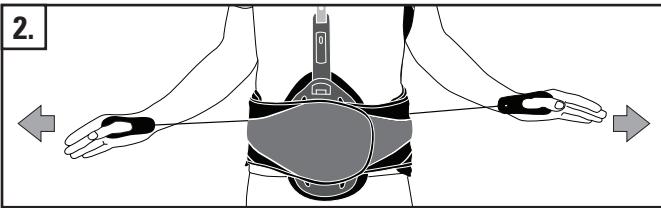
ENGLISH

TIGHTEN

Pull strongly away from body. Tighten more than you need and then loosen to achieve a proper fit. Pull left tab to tighten lower, pull right to tighten upper as needed.

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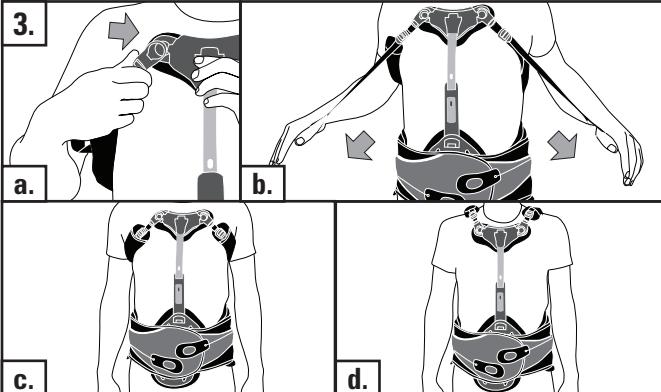
ENGLISH

STRAPS

- Attach underarm control straps to sternal pad.
 - Pull underarm control straps to tighten as needed.
 - Underarm control straps and sternal pad should fit securely to patient's anatomy.
 - Under arm control straps can be positioned to fit over the shoulder.
- Note: If needed, the support strut can be bent to ensure a proper fit.

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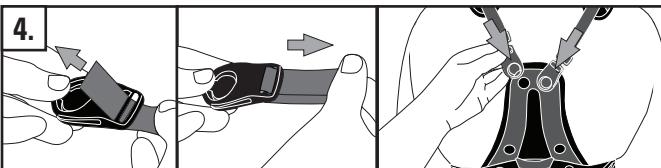
ENGLISH

OVER SHOULDER OPTION

If buckles are needed, add a buckle to strap ends and secure buckle. Attach buckles to the top of the TLSO back panel.

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ENGLISH

ANTERIOR PANEL EXTENSION

Position anterior panel extension pads on brace for desired coverage. Panels must overlap to encircle the trunk.

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LIMITED WARRANTY

Aspen Medical Products, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products in the event of breach of warranty shall be to repair or replace the defective product or parts.

Aspen Medical Products shall have no obligation under this limited warranty in the event:

- The product was not purchased from Aspen Medical Products or through its authorized channels of distribution;
- The product was altered;
- Any parts not supplied by Aspen Medical Products are inserted into the product; or
- The product is not used in accordance with the Aspen Medical Products, Instructions for Use.

THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF, OR DEFECT IN, THE PRODUCT. ASPEN MEDICAL PRODUCTS SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS.

This warranty gives you specific legal rights and you may have other rights which vary from state to state. Warning: To ensure proper use, please review all material. A listing of authorized distributors in your area is available upon request.

READ INSTRUCTIONS BEFORE USE.

Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations.

THIS PRODUCT IS INTENDED FOR APPLICATION BY LICENSED HEALTH CARE PRACTITIONERS AS DIRECTED BY A PHYSICIAN OR OTHER QUALIFIED MEDICAL AUTHORITY. PERMANENT INJURY MAY RESULT FROM USE OTHER THAN AS DIRECTED BY A PHYSICIAN OR OTHER MEDICAL AUTHORITY. ASPEN MEDICAL PRODUCTS SPECIFICALLY DISCLAIMS ANY OBLIGATION OR LIABILITY FOR INJURY OR DAMAGE BY REASON OF ANY APPLICATION OF THIS PRODUCT. PLEASE SEE THE ACCOMPANYING LIMITED WARRANTY FOR FURTHER INFORMATION IN THIS REGARD.