# PROVOX<sup>®</sup>

## Vega<sup>™</sup> Plug








CE 0413






# Atos

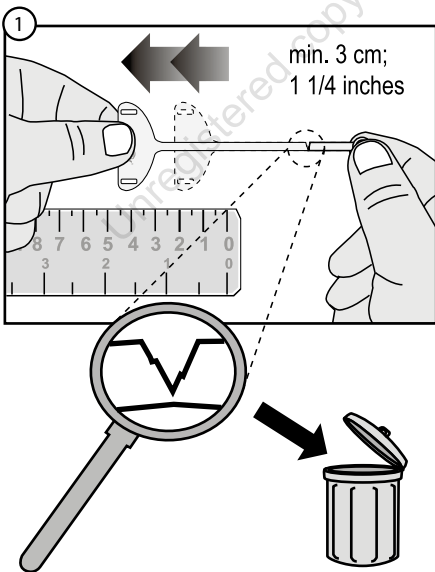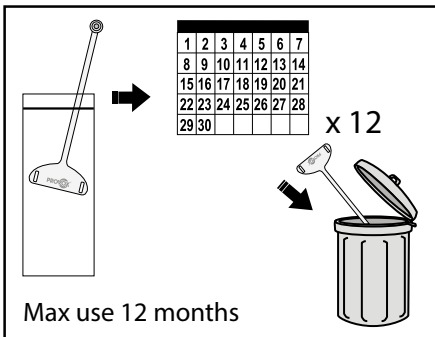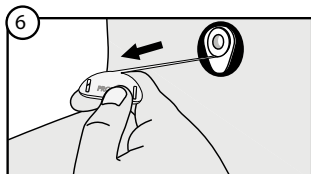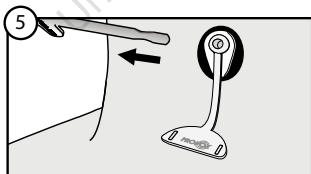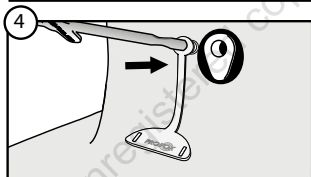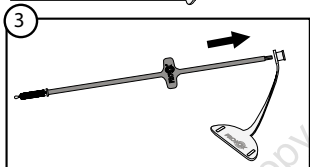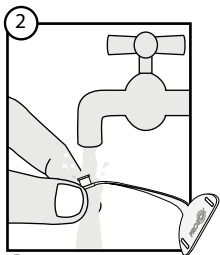Atos Medical **Your voice**

2 / 250 ml (8,5 fl oz)
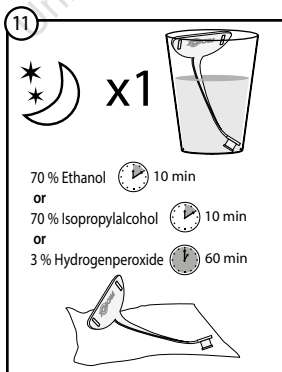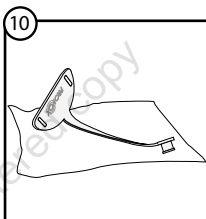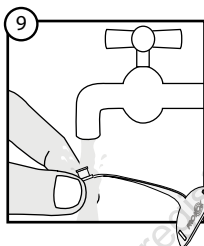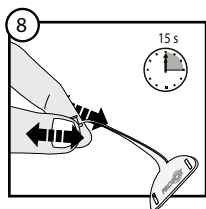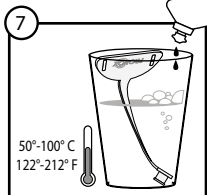
## **Disclaimer**

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

## **Patents and trademarks**

The Provox<sup>®</sup> System is protected by several patents. Provox<sup>®</sup> is a registered trademark owned by Atos Medical AB, Sweden.

Unregistered copy

# Contents

ENGLISH .....	7
DEUTSCH .....	8
NEDERLANDS .....	10
FRANÇAIS .....	11
ITALIANO .....	12
ESPAÑOL .....	13
PORTUGUÊS .....	14
SVENSKA .....	15
DANSK .....	16
NORSK .....	17
SUOMI .....	18
POLSKI .....	20
ΕΛΛΗΝΙΚΑ .....	21
РУССКИЙ .....	23
TÜRKÇE .....	25
日本語 .....	26
한국어 .....	27
简体中文 .....	28
עברית .....	29
العربية .....	30
ORDERING INFORMATION .....	31

The Instructions for Use, which accompanies this product, may be revised from time to time and must therefore be reviewed prior to each procedure in which the product is used.

The Provox Vega Plug is intended to be used with Provox Vega voice prostheses only.

Check the device integrity prior to each use by pulling the strap (Fig. 1).

## Cleaning and disinfection

Clean the Provox Vega Plug after each use, or if it is removed during use according to Fig. 7-10.

**DO NOT** use any water other than drinking water to clean and rinse the device.

Disinfect the Provox Vega Plug at least once a day if it is removed (Fig. 11) with one of the following methods:

- Ethanol 70% for 10 minutes
- Isopropylalcohol 70% for 10 minutes
- Hydrogenperoxide 3% for 60 minutes

**CAUTION: DO NOT** clean or disinfect by any other method as this might cause product damage and injury.

If the Provox Vega Plug looks dirty or has air dried in an area with a risk of contamination the device should be both cleaned and disinfected before use. A risk of contamination could exist if the device has been dropped on the floor or if it has been in contact with a pet, someone with a respiratory infection, or any other gross contamination.

During hospitalization it is important to both clean and disinfect the Provox Vega Plug after use but also **before** use since there is an increased risk of infection and contamination. In a hospital it is better to use sterile water for cleaning and rinsing, rather than drinking water.

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© Atos Medical AB, 2016.