# 4030 Manu Direxa Basic, 4032 Manu Direxa Basic W&T

GB

## INFORMATION

Please read the following directions carefully and pay special attention to the safety information!

### 1. Description

#### 1.1 Intended Use

Art. 4030 Manu Direxa Basic and art. 4032 Manu Direxa Basic W&T are to be used exclusively for orthotic treatment of the wrist respectively wrist and thumb. They are exclusively suitable for contact with healthy skin.

#### 1.2 Indications and Effects

**4030:** Acute and chronic wrist irritations, sprains, tendinopathies, osteoarthritis, carpal tunnel syndrome, weakness and pain in the wrist.

**4032:** Acute and chronic wrist irritations, sprains, tendinopathies, Quervain's disease, osteoarthritis, rhizarthritis, carpal tunnel syndrome, weakness and pain in the wrist and thumb.

The orthosis acts stabilizing and pain relieving.

Indication must be determined by the physician.

#### 1.3 Contraindication

##### 1.3.1 Relative contraindication

In case of the following indications, consultation with a physician is required: Skin diseases/ -injuries, inflammatory symptoms, prominent scars that are swollen, reddened and feel excessively warm when touched; Lymphatic flow disorders, including unclear soft tissue swelling distant to the body area to which the appliance will be applied; Sensory and circulatory disorders of the arm and wrist.

#### 1.4 Safety Instructions

## INFORMATION

The patient is to be instructed in the proper use/care of the product. The initial fitting and application of the product must only be carried out by trained, qualified personnel. The daily duration of use and period of application are dependent on medical indication by the physician.

## CAUTION

**Risk of injury as a result of improper use.** The product is designed for use on one patient only. Consult a physician immediately if you experience unusual changes (such as increase in pain). Improper use changes to the product are not permitted. An orthosis/support applied too tightly to the body can cause local pressure and, in some cases, even restrict adjacent vessels or nerves.

## NOTICE

**Damage due to incorrect environmental conditions.** This product is not flame-resistant. Keep the product away from flames or other heat sources. The product should not come into contact with grease or acidic agents, unguents and lotions.

### 2. Handling

#### 2.1 Size Selection of the product

The orthosis size is selected according to the wrist circumference.

#### 2.2 Adapting and Application of the product

The initial fitting and application of Manu Direxa Basic and Manu Direxa Basic W&T may only be carried out only by qualified personnel.

- Place the support on the palm side of the wrist.
- Close the Velcro straps individually, start with the outermost straps, and complete the closure by closing the middle strap.
- Make sure the brace is firmly fitted without too much pressure.
- If necessary adapt the splint.

#### 2.3 Instruction for use and care

**Material:** Lycra®, rayon, polyester, nylon. Metal splint.

The product is not flameproof; avoid temperature above 120°/248°F. **Disposal after use:** Combustible material.

**Cleaning:** Machine wash 40° Celsius, with a standard mild detergent, air-dry.

Note: Residues of cleaning agents may cause skin irritations and wear of material.

To extend the products life-time: close all Velcro® when not using the product.

### 3. Further Usage Restriction Clause

The product has been designed for use on only one patient. The daily

duration of use and period of application are dependent on the medical indication.

Latex Safe: To our knowledge this products does not contain any natural rubber.

#### 4. Liability

The manufacturer's warranty applies only if the device has been used under the conditions and for the purposes described. The manufacturer recommends that the device be used and maintained according to the instructions for use.

#### 5. CE Conformity

This device meets the requirements of the 93/42/EEC guidelines for medical devices. This device has been classified as a class I device according to the classification criteria outlined in appendix IX of the guidelines. The declaration of conformity was therefore created by Otto Bock with sole responsibility according to appendix VII of the guidelines.

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