DOLOMITE® Futura™ / Symphony™
Rollator

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User Manual

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DE

FR

GB

IT

NL

CE

DOLOMITE

Yes, you can.®

Bedienungsanleitung DE	2
Manuel d'utilisation FR	10
User Manual GB	20
Istruzioni per l'uso IT	26
Gebruiksaanwijzing NL	32

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User Manual

1 GENERAL	19
General Information.....	19
Symbols	19
Product Life Cycle	19
Product Labeling.....	19
User Height Specifications and Weight Limitation	19
2 SAFETY	20
Intended Use.....	20
General Warnings.....	20
3 USAGE	21
Operating Position.....	21
Folding the Rollator.....	22
Adjusting the Handle Height	22
Adjustment of the Braking Effect	23
Using the Hand Brake.....	24
Using the Parking Brake	24
4 CARE AND MAINTENANCE	24
5 AFTER USE	25
Disposal.....	25
Reuse	25
6 WARRANTY	25
Warranty	25

I GENERAL

I.1 General Information

Study the operating instructions carefully before use. The person who prescribes the product or who is otherwise responsible for the user must ensure that the product is suitable for the user.

I.2 Symbols

Signal words are used in this manual to apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

WARNING

Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

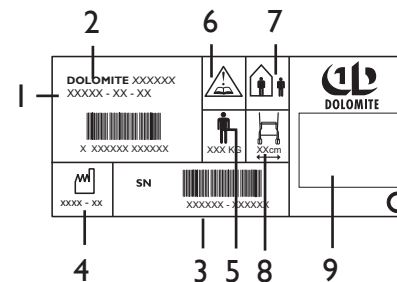
Gives useful tips, recommendations and information for efficient, trouble-free use.

I.3 Product Life Cycle

The expected product life cycle of this product is 5 years when used in accordance with safety instructions, maintenance intervals and correct use, stated in this manual. The effective product life cycle can vary according to frequency and intensity of use.

I.4 Product Labeling

- 1: Article number
- 2: Product name
- 3: Serial number
- 4: Date of manufacture
- 5: Max. user weight
- 6: NB! See user manual.
- 7: In- and outdoor rollator.
- 8: Max. width of rollator.
- 9: Manufacturer's name.


I.5 User Height Specifications and Weight Limitation

N.B. Your walker should only be used as a walking aid!
User's max. weight: **Futura** 150 kg, **Symphony** 100 kg.

INVACARE will not accept responsibility for adaptations, faulty repairs, etc, that are outside their control. *

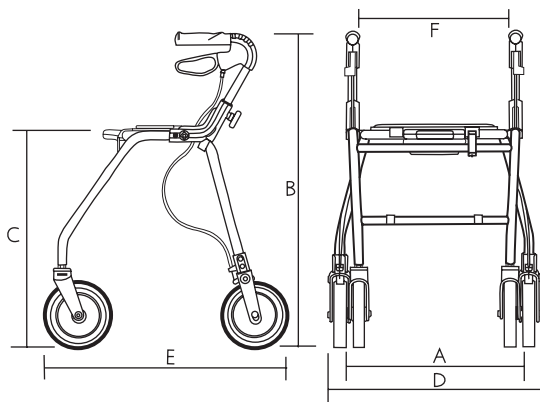* The weight limitation includes the contents of the basket, tray or shelf and the weight of the user whether standing or sitting.

Dimensions Futura	Measurement			
	680	600	520	450
A	500	470	470	460
B	860-1015	790-930	710-850	640-790
C	705	620	545	475
D	600	600	600	600
E	780	700	660	590
F	420	420	420	420

Dimensions Symphony	Measurement		
	600	520	450
A	460	460	460
B	785-935	705-850	640-790
C	625	540	480
D	600	600	600
E	710	645	620
F	420	420	420

2 SAFETY

- ⓘ Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

2.1 Intended Use

The rollator is intended to be used solely as a walking aid and all other use can cause risk of injury.

2.2 General Warnings

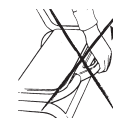
WARNING

Exercise caution when walking on sloping ground.

WARNING

Never lift the rollator in the folding handle when it is in use.

WARNING

Not to be used for standing on or as a wheelchair.

WARNING

Do not go up or down stairs with the rollator.
Do not use the rollator on escalators.
Do not under any circumstances stand on the walker.
If the push handles or the seat are exposed to extreme temperature (above 41 °C) there might be risk for first degree burns.
The walker must be fully unfolded when used as walking support
Use seat for stationary seating only, note that the parking brake must be engaged. Never sit and push with your feet.
The handlebar must not be angled outwards as this reduces the rollator's lateral stability.
The parking brakes must be in braked position when the rollator is used as a support when getting up, from a chair for example.
Loads must only be transported in the basket or on the tray.

3 USAGE

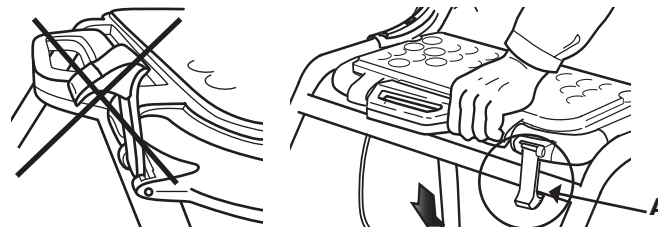
3.1 Operating Position

For walking position: Hold the rollator in the push handles, press downwards on the push handles when you have all four wheels on the ground, alt. press the seat downwards. (see picture). A clear sound indicates the correct position.


WARNING

Always check that the safety latch (A) is in locked position.

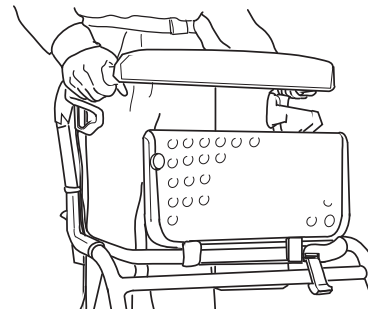
The seat can be folded up to provide a larger walking space, and in order to increase accessibility.

3.2 Folding the Rollator

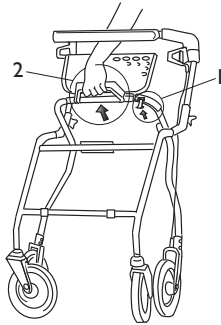1. Lift up/loosen the safety latch.
2. Lift the handles in order to fold the walker into folded position.


3.3 Adjusting the handle height

For adjustment; Loosen the wing, adjust the handle to suitable height and tighten the wing properly. When standing behind the walker with flat shoulders adjust handles to be at the same height as the user's wrists. Therapist may suggest different heights depending on specific need of the client. If the user unconsciously tends to adjust the height adjustment handles, for safety reason the handles should be tightened with a socket head wrench.


WARNING: Risk of falling

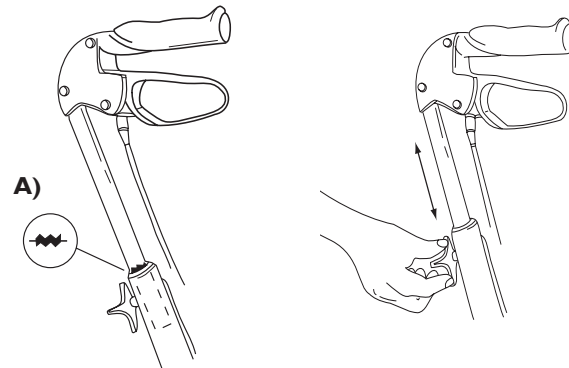The handles must never be set above their highest position, indicated by the marking on the tubes (see picture A).

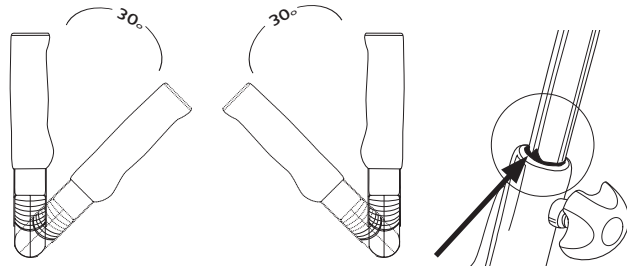
For optimum performance insure that handles are in the allowed adjustment area at all times. The arrow, which is placed on the tubes on the inside of the rollator, pointing on the thin inner vertical line indicates maximum allowed outer handle position.

3.4 Adjustment of the Braking effect

For desirable braking strength, adjust the brake wire with suitable tools (not included). The wire can be adjusted in two places - above under the brake handle or below above the brake clamp. The brake handle must be in neutral position.

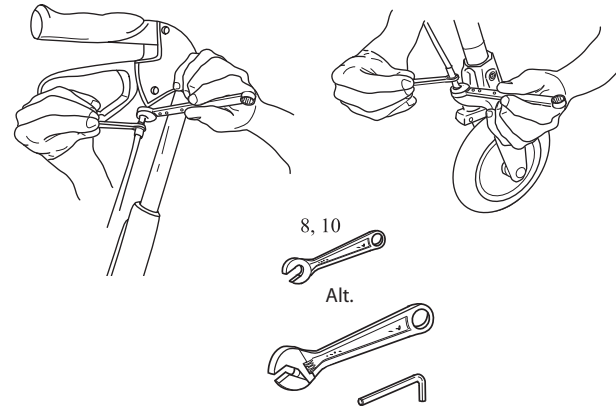Loosen the locking nut and turn the hexagon nipple counter clockwise in order to increase the braking effect. The distance between the brake clamp and the wheel should be 2 mm. Tighten the locking nut again to fixate the new adjustment. Check the braking effect by engaging the parking brake and check that the wheel can't rotate.


WARNING

Make sure to tighten the lock nuts after adjustment.

3.5 Using the handbrake

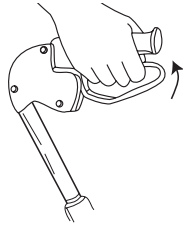Pull both brakes upwards to brake.


3.6 Using the Parking Brake

Press down carefully for parking brake. A clear sound is heard when the brake is placed in correct position.

- ⓘ Always place both brakes in parking position.

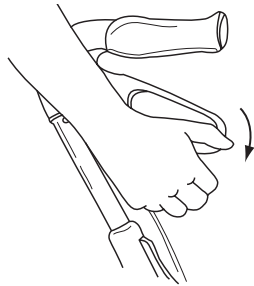Pull upwards to loosen the brake.

4 CARE AND MAINTENANCE

- ⓘ Maintenance should be done at least every six months.
Do not use an abrasive detergent, cleaner or cloth to clean or dry the walker.

To ensure good performance and safety you should regularly check that the following is carried out:

- Clean the rollator with solvent-free household cleaner. Keep the wheels clean.
- Ensure that screws and adjustments are properly tightened.
- Immediately contact the nearest dealer if a fault should appear.
- Never use a defective walker. Do not attempt repairs yourself!
- Verify brake operation. If brake adjustment is required, see section 3.6 in the user manual.
- If hand grips are loose Do not use the rollator. Change hand grips according to the service instructions which can be obtained from your Invacare representative.
- Periodically inspect the castors tightness, wear and damage.
- Make sure that the wheels are free of hair, lint and other debris.

5 AFTER USE

5.1 Disposal

When the product can no longer be used, it and its accessories as well as the packaging material should be sorted in the correct way. The individual parts can be separated and handled according to the list specification.

Please be environmentally responsible and recycle this product through your recycling facility at its end of life.

5.2 After Use

WARNING: Risk of Injury

Always inspect the product according to the service instructions before transferring the product to a different user. If any damage is found Do not use the product. Otherwise injury may occur.

The service instructions can be obtained from your Invacare representative.

6 WARRANTY

We supply a 2 year guarantee ** from the date of delivery. Damage caused by wear to items such as wheels, handles and brakes is not covered by the guarantee. Damage that arises due to abnormal use is excluded. Damage caused by users who weigh more than 150 kg (**Futura**), 100 kg (**Symphony**) is excluded. Maintenance and operating instructions must be observed for the guarantee to be valid.

 ** Deviations may occur due to local agreements.

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Invacare® Austria GmbH,
Invacare® AG,
Invacare Rea AB