

Hydrofera Blue classic foam



WOUND CARE DRESSING APPLICATION GUIDE

Hydrate




- Hydrate with sterile saline or sterile water
- Squeeze out excess fluid

Apply




- Dressing may be cut to fit the wound bed
- If rolled edges are present, the dressing should cover both the wound and periwound area

Cover

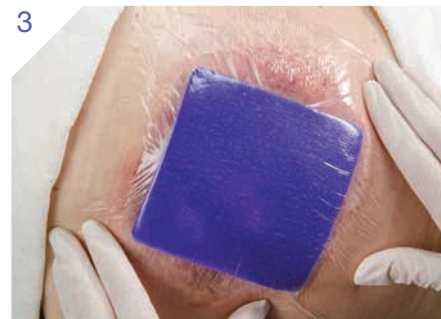

- Secondary dressing selection is based on exudate level (see chart below)

Dressing Changes

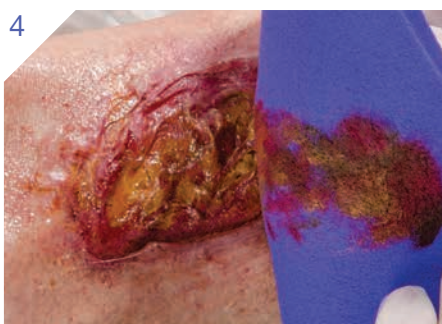


- Change the first dressing at 24 hours
 - If the dressing retained its blue color where it had been in contact with the wound, the new dressing can be left in place for up to 72 hours
 - If the dressing turned white or lightened in color, continue 24-hour dressing changes until blue color is retained

Moisture Balance



- As with many wound care dressings, moisture balance is key. If the dressing is dry, thoroughly remoisten with sterile saline or water, then gently remove the dressing.

Secondary Dressing Selection Guide

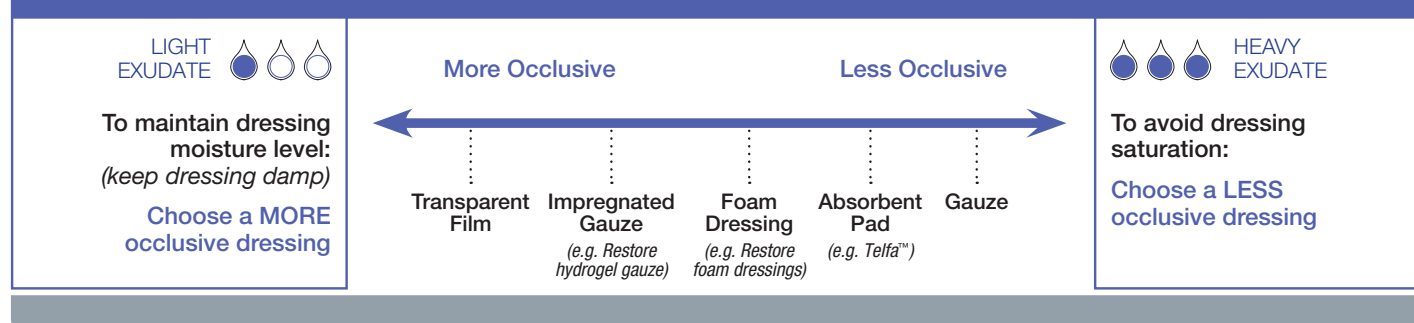

Hydrofera Blue classic foam Ordering Information

Stock No	Product Size	Quantity/Box	HCPCS*
Standard Dressings			
HB2214	2" x 2" (5 cm x 5 cm)	10	A6209
HB4414	4" x 4" (10 cm x 10 cm)	10	A6209
HB6614	6" x 6" (15 cm x 15 cm)	10	A6210
HBT0906	9-mm Tunneling Dressing (1.2 g)	10 pair	A6215
Heavy Drainage Dressings			
HBHD4450	4" x 4" x .5" thick (10 cm x 10 cm x 1.2 cm)	5	A6209
HBHD6675	6" x 6" x .75" thick (15 cm x 15 cm x 1.9 cm)	5	A6210
Dressings With Moisture-Retentive Film			
HBRF4414	4" x 4" (10 cm x 10 cm)	10	A6209
HBRF2258	2.25" x 8" (5.7 cm x 20 cm)	10	A6210
HBRF4475	4" x 4.75" (10 cm x 12 cm) Island Dressing; pad size 2" x 2.75" (5 cm x 6.5 cm)	10	A6212
HBRF2650	2.5" (6 cm) Diameter Ostomy Dressing	10	A6209


For more information, see product instructions included with the product.

Additional Instructions

Manufactured for:
Hollister Incorporated
2000 Hollister Drive
Libertyville, IL 60048 USA

Distributed in Canada by:
Distribué au Canada par:
Hollister Limited
95 Mary Street
Aurora, Ontario L4G 1G3

www.hollisterwoundcare.com

 Caution: Federal (USA) law restricts this device for sale by or on the order of a physician or licensed healthcare professional. Refer to Instructions for Use for contraindications, warnings, precautions, and possible complications.

For product questions or detailed clinical questions concerning our products in the US call **1.888.740.8999**. In Canada call **1.800.263.7400**.

*HCPCS are for reference only and subject to change.

The Hollister logo is a trademark of Hollister Incorporated. Hydrofera Blue and the Hydrofera Blue logo are trademarks of Hydrofera, LLC.

©2014 Hollister Incorporated.

Made and printed in the USA.

922615-0514

