

Application instructions **Men's Liberty™ Acute external catheter**



1. Clean

- **Always begin with a clean and dry penis**
- Keep foreskin back completely throughout application
- To begin, wipe the head of the penis with the alcohol wipe and **allow to dry**



Tip for success: Retract foreskin on patients that are uncircumcised to apply

2. Prep skin


- Wipe the head of the penis with BioPlus™ wipe
- **Allow 15–20 seconds to dry until the skin is no longer tacky**



Tip for success: If BioPlus+ is not completely dry the product will not stick

3. Positioning & application


- Remove release papers around the petals
- Attach the faceplate to the tip of the penis with the **center opening directly in line with the urethra**
- Smooth down petals

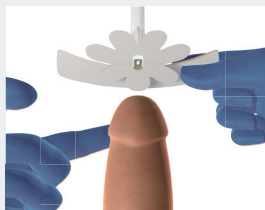

Tip for success: Line up the opening with the urethra to prevent leaking


4. Seal

- Apply the faceplate strip around the head of the penis, flat and wrinkle free
- Press and hold for 10–15 seconds to secure the seal

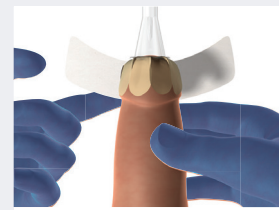

Tip for success: Ensure the wings cover the gaps between petals to prevent leaking

5. You have completed MLA application

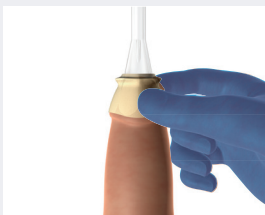

6. Connect to leg or bedbag


- Connect to leg or bedside bag
- Be sure to hang below the bladder or bed to ensure drainage



Tip for success: Do not connect to suction

7. Secure tubing

- For best results, use CathGrip® UniGrip™ to secure bag and reduce tension

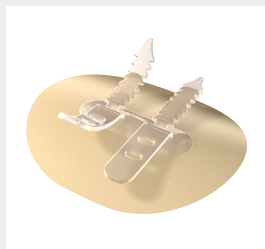

Wear time

- **The first application for each patient will be shorter than the subsequent applications**
- This is due to the hydrocolloid material that absorbs underlying moisture in the skin



First application



Second application

Removal

Remove and replace MLA when:

- hydrocolloid turns white color
- loose
- soiled

Then we recommend using FreeDerm because it dries quickly and does not leave an oily residue

