


## AEROCHAMBER PLUS® Z STAT® ANTI-STATIC VALVED HOLDING CHAMBER (aVHC)

### Quick Start Guide: Instructions for Use


**1** Remove MDI cap.

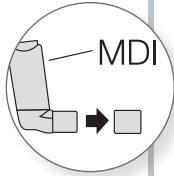


**2** Shake MDI.




**3** Insert MDI into chamber.

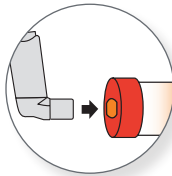

**4** Apply mask to face.

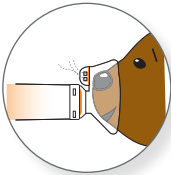

**5** Press MDI and count for 5 to 6 breaths.

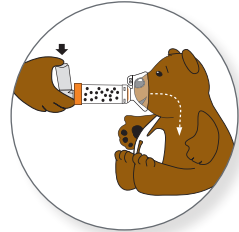

**6** Repeat steps 2 to 5 as prescribed.




Small Mask

Medium Mask

Mouth-piece

Large Mask

These are abbreviated instructions for use, please remember to read the complete instructions that are packaged with your AEROCHAMBER PLUS® Z STAT® anti-static Valved Holding Chamber (aVHC) device.

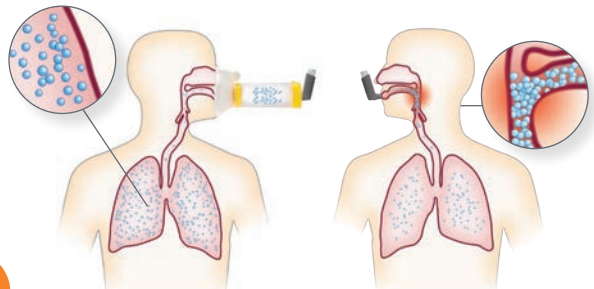

**Using Inhaler with aVHC:**  
Medication reaches lungs

**Using Inhaler Alone:**  
More medication collects in mouth and throat (potential for side effects)



## AEROCHAMBER PLUS® Z STAT® ANTI-STATIC VALVED HOLDING CHAMBER (aVHC)



Small Mask


Medium Mask

Mouth-piece

Large Mask


### Quick Start Guide: Cleaning Instructions

**1** Remove backpiece only.

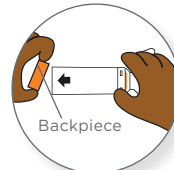


Backpiece

**2** Soak 15 minutes in soapy water, agitate gently and rinse or place in dishwasher on top rack.

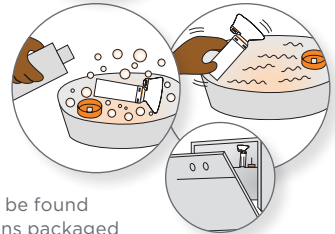


Dishwasher instructions can be found with the complete instructions packaged with your AEROCHAMBER PLUS® Z-STAT® aVHC.

**3** Shake out water and dry vertically.

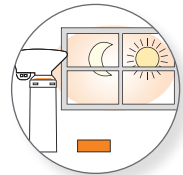

**4** Reassemble backpiece.

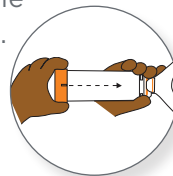

These are abbreviated cleaning instructions, please remember to read the complete instructions that are packaged with your AEROCHAMBER PLUS® Z STAT® anti-static Valved Holding Chamber (aVHC) device.



5 Latour Avenue  
Suite 1600  
Plattsburgh, NY 12901

518.561.7330 **phone**  
800.833.9653 **toll free**  
518.561.5088 **fax**

monaghanmed.com



Words or phrases accompanied by ™ and ® are trademarks and registered trademarks of Monaghan Medical Corporation or an affiliate of Monaghan Medical Corporation. © 2015 Monaghan Medical Corporation. † trademarks and registered trademarks of the respective companies.

P/N 79113-03, 11/2016