### ENGLISH

#### FIGURE 1

- A. Ankle Cuff
- B. Ankle Cuff Buckle
- C. Ankle Clamp
- D. Nut and Bolt
- E. Shoe Inlay
- F. Connection Strap
- G. Connection Strap Keyhole
- H. Optional Foot Wrap

#### INDICATIONS FOR USE

Drop foot

#### PRECAUTIONS

- Not intended for use by patients with spasticity.
- Never wear the brace over an open wound.
- Users with sensitive skin or those at risk of skin ulcerations and infections, such as in the case of diabetic peripheral vascular disease, are advised to consult with their practitioner before use of this product.
- Do not over-tighten braces as risk loss of blood flow.
- If the user experiences pain, swelling, sensation changes, or shows signs of insufficient blood flow (turning blue, white, or cold) while using the brace, advised to discontinue use immediately and consult practitioner.

#### INITIAL APPLICATION AND ADJUSTMENT

##### Donning Ankle Cuff

- Measure ankle circumference with ankle cuff (Figure 2).
- The user is advised against donning tight shoes or socks over the Foot Wrap, as this may result in diminished blood flow to/from the foot causing injury.
- For users with lower-leg width in the lowest range, the ankle clamp may not provide an optimal range of hold when donning the device.

##### Nut and Bolt (Figure 5)

- Ensure a secure and tight fit between nut and bolt.
- Bolt can be screwed with a coin if required.
- Nut fit should be checked periodically and tightened if required.

##### Shoe Attachment (for medial or lateral configuration)

- Thread bolt through shoelace eyelet on either side of the shoe (ideally towards the toe-end) for increased medial or lateral control (Figure 6A).
- Fasten bolt with nut securely (Figure 7A).
- Snap keyhole into place over nut (Figure 8A).
- Thread connection strap through ankle cuff buckle, and pull to tighten (Figure 9A).

##### WASHING & CARE

#### LIST OF MATERIALS

Nylon, polyethylene, polypropylene, cotton, polyurethane, polyester, silicone (printing).

#### WARRANTY

Ossur provides a limited 6-month warranty.

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