Part #	D491
Revision	B
Description	EZCare Softouch DFU
Artwork #	AW4491
Material # (PM, DM, LM) & description	DM466 50# opaque smooth offset White
Effective date	
CO #	

**NEOTECH™**  
Products, Inc.  
**EZCare™ Softouch**  
**Tracheostomy Tube Holder**

REF Catalog No.	DIRECTIONS FOR USE INSTRUCTIONS D'UTILISATION NÁVOD K POUŽITÍ <sup>1</sup> GEBRAUCHSANWEISUNG ISTRUZIONI PER L'USO INSTRUCCIONES DE USO ZALECENIA DOTYCZĄCE UZYTKOWANIA
N9105 5 inch	
N9106 6 inch	
N9107 7 inch	
N9108 8 inch	
N9109 9 inch	
N9110 10 inch	
N9111 11 inch	
N926 18 inch 2-piece	

Manufactured by  
NEOTECH PRODUCTS, INC.  
28430 Witherspoon Pkwy, Valencia, CA 91355, USA  
1-800-966-0500 • neotechproducts.com

EMERGO EUROPE  
Molenstraat 15  
2513 BH, The Hague, The Netherlands  
Tel: +31.70.345.8570, Fax: +31.070.346.7229

D491 Rev B

**English**  
**EZCare™ Softouch**  
**Tracheostomy Tube Holder**

**FEATURES & BENEFITS**

- Thick NeoFoam® Ultra for better stability
- Soft, breathable, skin friendly NeoFoam wicks perspiration
- Designed for comfort
- Lightweight and form fitting
- N926 18" can be cut to size
- Unique material helps to prevent sliding
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

**INDICATIONS FOR USE**

The EZCare Softouch is intended to secure a patient's tracheostomy tube. It is intended for use on pediatric (neonates, infants and children) patients.

**DIRECTIONS FOR USE**

**Measuring**

Select proper size by using EZCare Softouch measuring tape strip provided.

Place start line at flange on left side and wrap behind neck to right flange. If measurement falls between sizes, try larger size first.

**Single Piece (N9105-N9111)**

**Step 1** With smooth, white side facing neck, insert hook tab through flange on one side of plate.

**Step 2** Pull tab back over flange and firmly press white hook fabric to neckband fabric to ensure proper securing.

**Step 3** Wrap EZCare around neck and insert hook tab through flange on other side of plate.

**Step 4** Pull tab back over flange and firmly press white hook fabric to neckband fabric to ensure proper securing.

Note: Ensure that EZCare is taught, but not overly stretched. Adjust as needed.

**Step 5** Replace EZCare Softouch every 5 days or per hospital protocol, whichever is sooner.

Replace immediately if hook tab is worn or visibly soiled.

**2-piece (N926)**

**Step 1** Separate two pieces.

**Step 2** With smooth, white side facing neck, insert hook tabs through flanges on each side of plate.

**Step 3** Pull tab back over flange and firmly press white hook fabric to neckband fabric to ensure proper securing.

**Step 4** Wrap both sides of EZCare around neck and bring together.

**Step 5** Determine proper length, pull longer piece away from neck and cut excess.

Note: Ensure that skin will not be exposed to hook fabric secure tab.

**Step 6** Bring both sides back together and firmly press white hook fabric secure tab to neckband fabric to ensure proper securing.

Note: Ensure that EZCare is taught, but not overly stretched. Adjust as needed.

**Step 7** Replace EZCare Softouch every 5 days or per hospital protocol, whichever is sooner.

Replace immediately if hook tab is worn or visibly soiled.

**CAUTIONS & WARNINGS**

Federal law restricts this device to sale by or on the order of a physician.

Replace immediately if hook tab is worn or visibly soiled. Worn tab may cause decannulation.

Check regularly to prevent disengagement.

Discontinue immediately if skin irritation occurs.

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