




Item# 567969 - Plate Rack &
567970 - Olympic Plate Rack

Parts:

- A-Triangle Frame (1)
- B-Round Plastic End Caps (7)
- C-Nuts (4)
- D-Washers (4)
- E-Base Frames (2)
- F-Bolts (4)
- G-Rectangular Plastic End Caps (4)

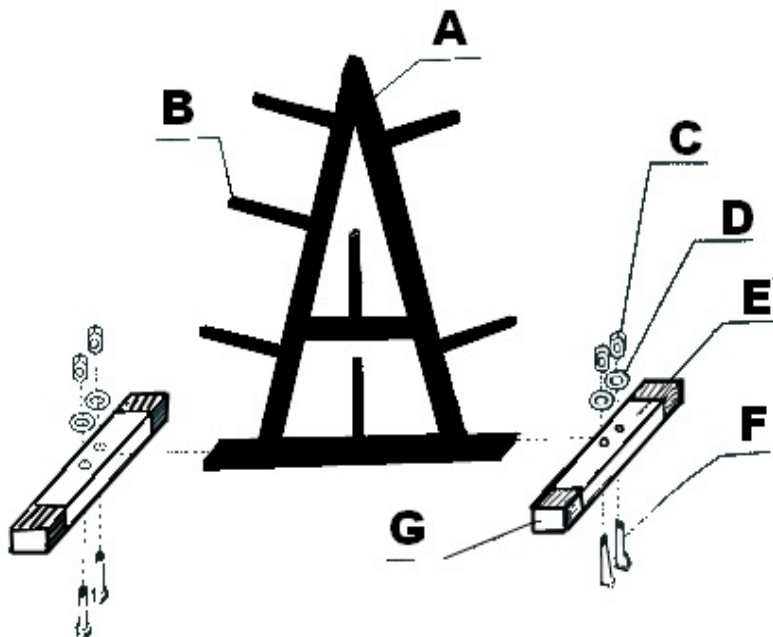

*****For assembly of this item you will need a wrench.**

Assembly Instructions:

- Attach the 2 base frames to the bottom of the triangle frame. Insert two bolts from bottom on both sides of the frame. Attach the washers and nuts on all four, use the wrench to secure tightly.
- Place round plastic end caps on all of the pegs.
- Place Rectangular plastic end caps on the ends of the base frames.

Cleaning Instructions:

Wipe rack with mild soap
and/or disinfectant.



Patterson Medical
1000 Remington Blvd.
Suite 210
Bolingbrook, IL 60440
Phone: 800-323-5547
Direct Phone: 630-378-6000
Fax: 800-547-4333
Direct Fax: 630-378-6310