

5200 ERGOSEAT®



A revolutionary 2 piece seating device for repositioning and standing assistance, ErgoSafe® proudly presents the ErgoSeat®. There are no restrictions to movement as the ErgoSeat® offers omnidirectional options, meaning the patient can move laterally, in a circular motion or prepare to go from sit to stand. The ErgoSeat® has adjustable straps and a durable buckle to easily secure to most seats or chairs.

22.5" x 21.5" (57 x 54.5 cm)

ERGOSAFE PRODUCTS
A PRISM MEDICAL COMPANY



5200 ERGOSEAT®

USE INSTRUCTIONS:

Place the ErgoSeat® with the shiny sides together and the softer sides to the outside. Secure the ErgoSeat® to the chair or seat using the strap and buckle.

Regular assessment of the patient's skin is essential if using the ErgoSeat® for extended periods of time. Consultation with a Registered Occupational or Physical Therapist is advised if the ErgoSeat® is begin considered for use with patients who have or are at risk of developing a pressure ulcer.

LAUNDRY INSTRUCTIONS:

The ErgoSeat® can be machine-washed at 80°C/170°F. Hang to dry or tumble dry on low heat at 40°C/104°F.

For more information on this and other ErgoSafe® products visit www.ergosafe-products.com

© 2009 Prism Medical. 45 Progress Parkway, Maryland Heights, MO. 63043

ERGOSAFE PRODUCTS
A PRISM MEDICAL COMPANY