
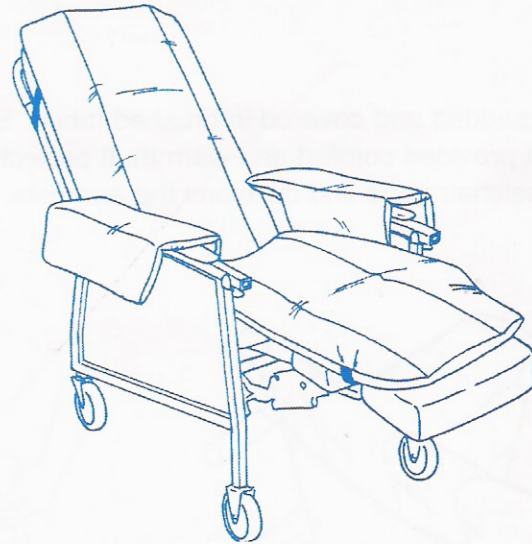


## GERI-CHAIR COZY SEAT

Reorder #	Description	Unit
703002	Geri-Chair Cozy Seat/Backrest and Seat	Each
703003	Geri-Chair Cozy Seat/Backrest, Seat, Leg Rest	Each

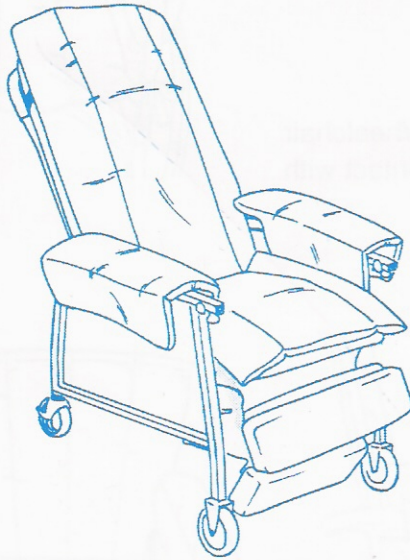



### Purpose

Skil-Care's Geri-Chair Cozy Seat provides comfort, pressure relief, and protection against bruises and skin tears. Designed to fit all styles of geri-chairs, the Cozy-Seat is reversible.

### Instructions for Use

The Geri-Chair Cozy Seat is available in two models: #703002 which covers the backrest and seat and #703003 which covers the backrest, seat, and leg rest. Both models are reversible. (Follow steps 1-3 for models 703002 and 703003.)

1. Place Cozy-Seat on geri-chair with label indicating "TOP" positioned at the top of the backrest.
2. Drape armrest cushions over armrests.
3. Secure straps behind backrest/ (Illustration A)
4. (For model 703003) Secure elastic band around leg-rest. (Illustration B)

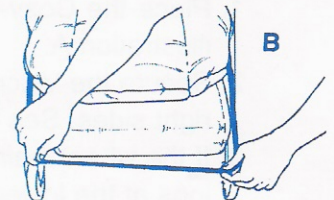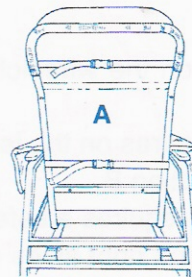

### Laundering

The Geri-Chair Cozy Seat is completely launderable. Do not wash or dry at temperatures that exceed 180°F (82°C). Do not use bleach.