

GERI-CHAIR LEG POSITIONER

Reorder #	Description	Unit
703420	Geri-Chair Leg Positioner	Each



Purpose

Skil-Care's Geri-Chair Leg Positioner is designed to comfortably position resident's legs on the extended geri-chair leg-rest. It prevents the legs from slipping off the sides of the leg-rest, and protects against bruises caused by contact with the chair frame.

Easy to Use

Place the Geri-Chair Leg Positioner on the extended leg-rest (anti-slip material facing down) and position it under the resident's ankles and lower calves. Wrap the strap around the leg-rest and secure the buckle. Tighten strap to prevent the leg positioner from shifting.

Easy Care

The durable nylon cover wipes clean with a damp cloth or sponge. A spray-type cleaner may be used. Do not launder.