

SleepSafe Manual/Electric

(HYBRID BED) ASSEMBLY GUIDE



Thank you for purchasing the SleepSafe® Safety Bed. With proper care, your bed will provide years of safe use. Carefully unpack all the contents of your bed and verify that all parts are included using the following pages.



we can help.
866-852-2337

PLEASE CHECK ALL BOXES FOR HARDWARE BEFORE DISCARDING
WARRANTY AND USER INFORMATION CAN BE FOUND AT THE END OF THESE INSTRUCTIONS

TOOLS NEEDED:



4mm Allen Wrench
(Provided)



Flat-Head Screwdriver



3/8" Wrench



At least 2 People

Table of Contents



3

Included Items

(Parts required to Assemble your Bed)

5

Bed Assembly Guide

(Detailed Instructions for Assembling your Bed)

9

Safety Rail Adjustment

(Up and Down)

10

Safety Rail Adjustment

(Side to Side)

11

Warranty Information

Included Items:



90mm Allen Bolt

x12



3/4" Hex Bolt

x8



Barrel Nut
(Preinstalled on padded beds)

x12



3mm Handle Bolt

x1



Handle Extension

x1



Cardboard Spacer

x4



1/4" Allen Wrench

x1



4mm Allen Wrench

x1



3mm Allen Wrench

x1



T15 Torx Wrench

x1



T10 Torx Wrench

x1

Included Items: (continued)



Casters

x4



Frame

x1



Crank Handle

x1



Mattress

x1



Headboard

x1



Footboard

x1



Side Rails

x2

Safety Rails


x2

Assembly Guide



1 Determine where your SleepSafe Bed® will be positioned:

Depending on the placement of your bed, it may be necessary to move the Crank Assembly to the opposite side of the bed. If so, set the frame flat on the ground, remove the Assembly using your two 7/16" wrenches, and reattach using the color-coded guide below.

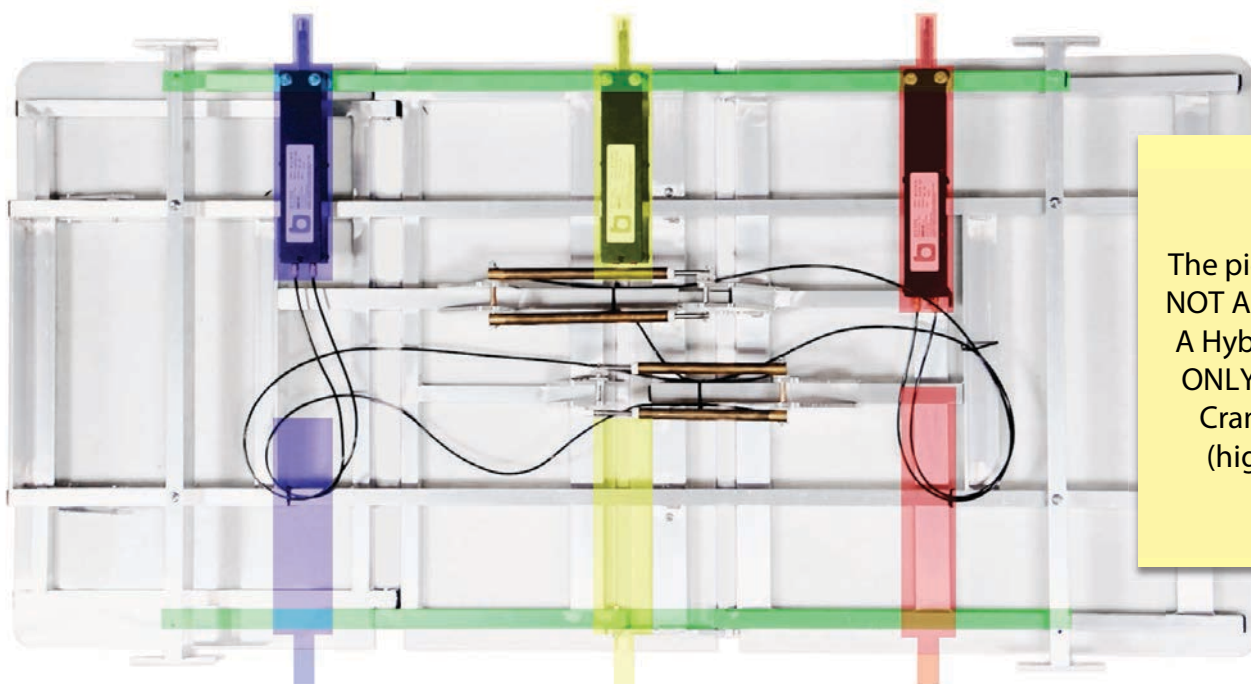

Note:

The picture shown is NOT A Hybrid Frame. A Hybrid Frame will ONLY have middle Crank Assembly (highlighted in yellow).

* Alternate Crank Assembly position should be a mirror image of its original position.

2 Set Frame on Cardboard Spacers:

One person should lift each side at a time while the other places the blocks underneath the foundation.



* Be sure to leave room to attach the side rails in the next step.

Assembly Guide (continued)




3 Attach Side Rails:



* The Side Rail with the holes goes on the side with the Crank Assembly


4 Attach Headboard and Footboard to Side Rails:

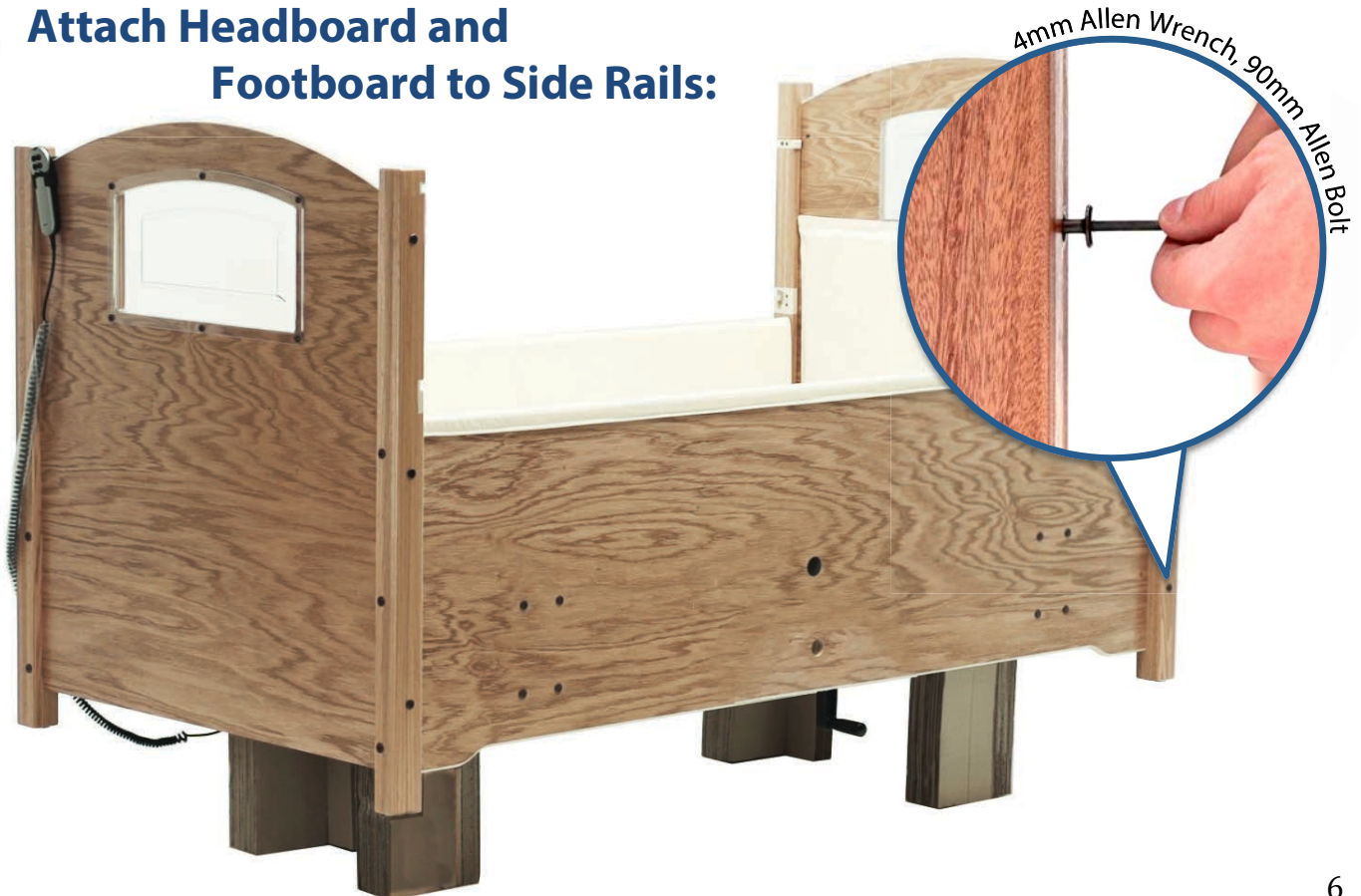

Assembly Guide (continued)



5 Install Casters and Remove Spacers

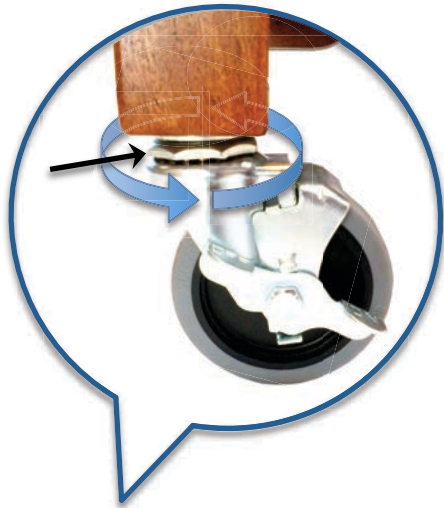

* Once the caster is inserted, turn the tightening nut as shown above to compress the rubber sleeve on the stem inside the wood leg, thus preventing the caster from falling out.

6 Install Mattress



Assembly Guide (continued)



7

Install Crank Handles:

Install Crank Handles in the following order:

1. Crank Extension
2. Crank Handle
3. 3mm Bolt



8

Install Safety Rails:



* Always install and remove Safety Rails while in their LOWERED position.

Safety Rail Adjustment

(UP AND DOWN)



*Each Safety Rail is pre-adjusted at our factory, but it may become necessary to adjust the Safety Rail height over time.

TOOLS NEEDED:



T10 Torx Wrench
(Provided)



T15 Torx Wrench
(Provided)



Rubber Mallet
(or Hammer)

1 Determine if your Safety Rails need adjustment:

Look at each brass locking-pin when the Safety Rail is in the locked position as indicated by "A" in Fig. 1. The pin should be centered in the hole of the plastic strike. If it is not, note which way the rail needs to move (up or down).

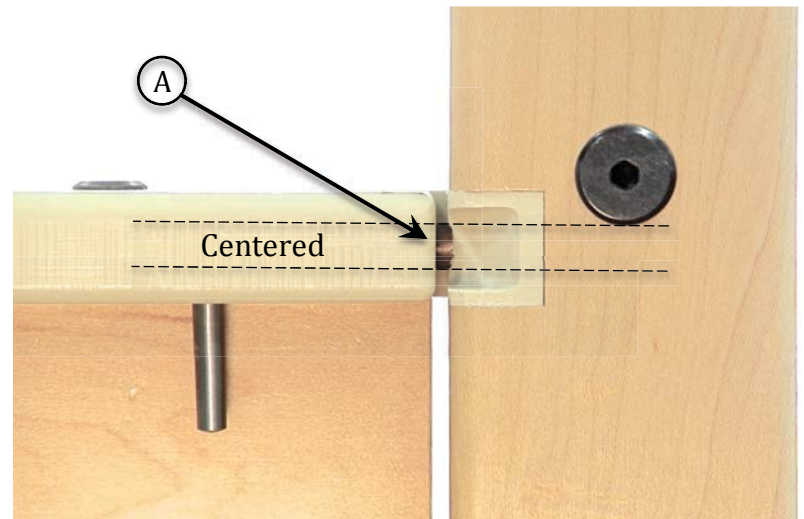


Fig. 1

2 Remove Plastic Strikes:

Use your T15 Torx Wrench to remove the gold screws holding your plastic strike in place. Then use a Rubber Mallet to remove the strike from the leg.

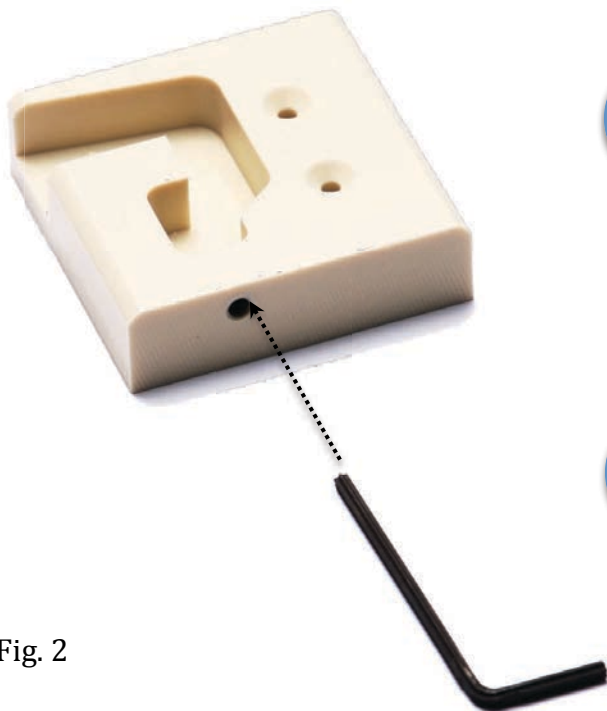


Fig. 2

3 Adjust Accordingly:

Insert the T10 Torx Wrench as shown in Fig. 2. Adjust the screw up or down depending on which way you noted in Step 1.

(Clockwise = Up / Counterclockwise = Down)

4 Reinsert Plastic Strike:

Place the Plastic Strike back into position and align by gently tapping it with your Rubber Mallet.

Safety Rail Adjustment

(SIDE TO SIDE)



*Each Safety Rail is pre-adjusted at our factory, but it may become necessary to adjust the Safety Rail height over time.

1 Determine if your Safety Rails need adjustment:


With the safety rail in the locked position, check the spacing between the end of the safety rail and the headboard and footboard legs.

The spacing should be even!




2 Remove Safety Rail and Adjust Accordingly:

If spacing is not even, or there is too much side-to-side play in the rail, remove safety rail and turn the black hex bolt using the $\frac{1}{4}$ " Allen Wrench provided.

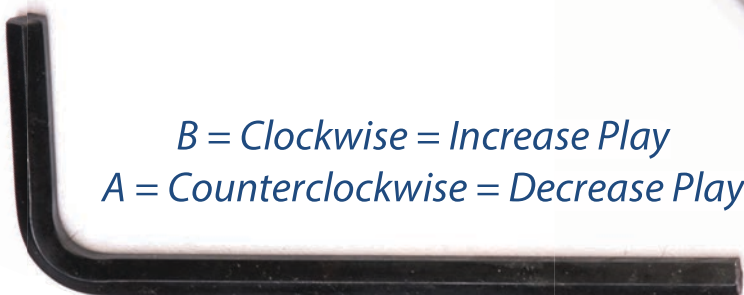

B = Clockwise = Increase Play
A = Counterclockwise = Decrease Play

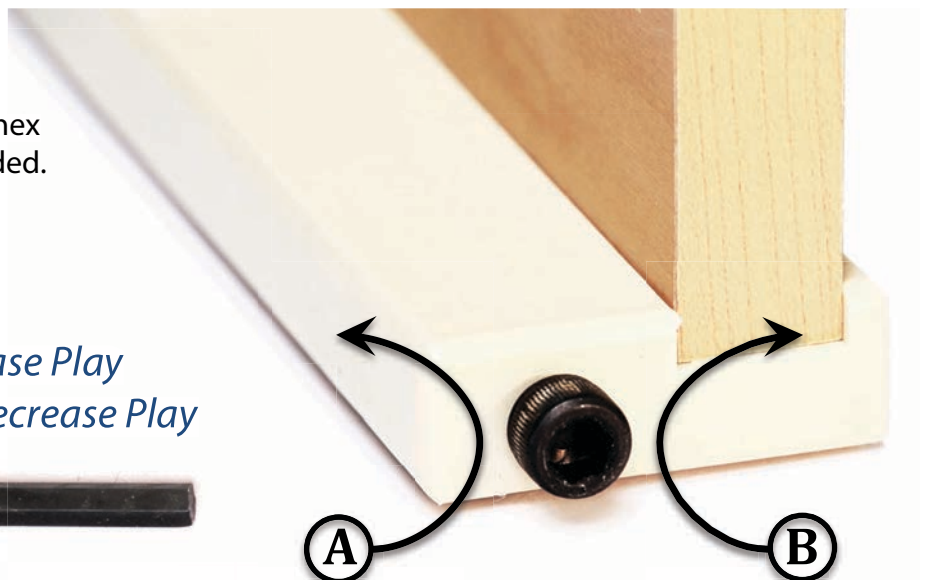

The Articulated Remote



**Chair Function
(Head & Foot)**
(Up/Down)

Head
(Up/Down)

Foot
(Up/Down)

REMOTE LOCK OUT PROCEDURE: PRESS and HOLD BOTH DOWN BUTTONS on HEAD and FOOT articulation functions for about 5 seconds. The light on the remote will flash when it is locked. To UNLOCK repeat this procedure.

Warranty Information



Full One-Year Warranty:

The SleepSafe™ Bed is warranted against defects in workmanship or materials for a period of one year from the date of purchase. SleepSafe Beds will repair any defective part at no cost to the purchaser.



Limited Five-Year Warranty:

During the second through fifth years from the date of purchase, SleepSafe Beds will replace any part found to be defective. Purchaser shall pay any service and shipping costs related to the replacement of the defective part.



Terms and Conditions:

This warranty applies to normal use and does not cover any damage caused by normal wear, abuse, misuse, mishandling or modification of the product.

*Safety Rails are intended to hinder or discourage the possibility of a roll-out. This product does not take the place of proper client supervision and monitoring. Any modification to the product or its components will void the warranty and render the product incapable of fulfilling its intended purpose.