

TheraFit Premium Soft Goods

NEW THERAFIT Y-HARNESS



Features:

- Use across the body to control shoulder rotation on one side
- Asymmetrical design for use on either shoulder
- Comfortable fit for both male and female
- **Therafin's exclusive hinged buckles help straps stay secure at various angles**
- Pull finger loops to adjust straps
- Hardware to attach to chair included

Description	Part Number
Small	30683
Medium	30684
Large	30685



All Finger Loop strap sections on this page are 31" long, without attaching hardware.

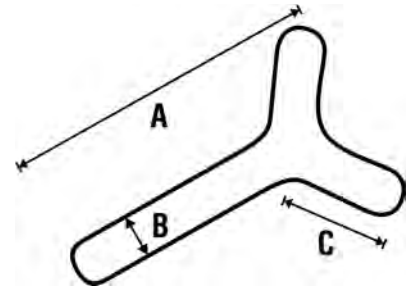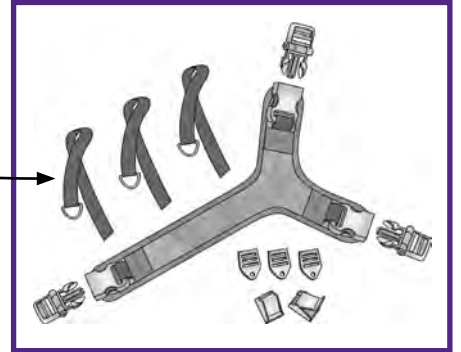

Harness Dimensions (inches)	A	B	C
SMALL	18	2	7
MEDIUM	20	2	8
LARGE	24	2.5	10



A harness is not a safety device. It is to be used only for positioning.

CUSTOM SEATING

BODY POSITIONING

WHEELCHAIR TRAYS

ASSISTIVE TECHNOLOGY

PATIENT TRANSFER



Attaching Hardware Information on page 68



Replacement Parts on page 92




STRAP POSITIONING GUIDES KEEP STRAPS FROM FALLING OFF SHOULDERS, PAGE 69

HARNESS

PARTS DESCRIPTION

- ◆ All parts shown here are not included with every harness
- ◆ Parts included vary depending on which harness you have

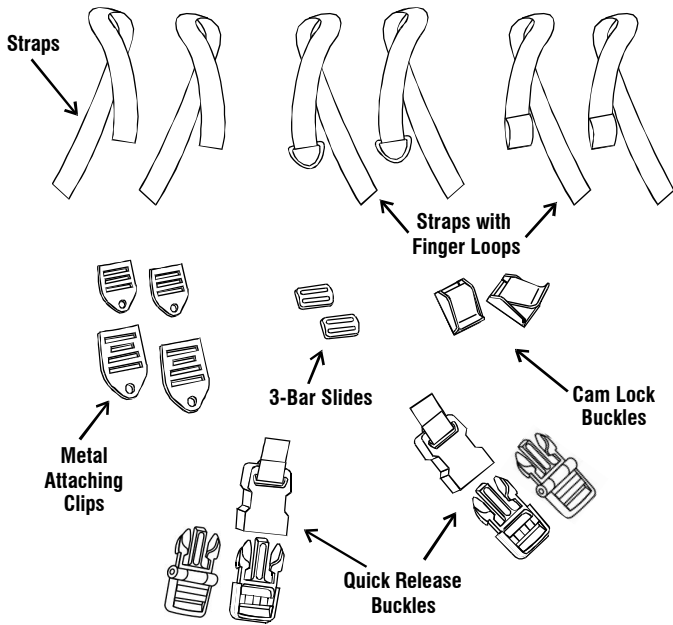


This harness is a **Kit** that you assemble as needed for your particular application. This sheet illustrates **examples** of the many ways to use the various straps, clips, and buckles.

HOW TO ATTACH A QUICK RELEASE BUCKLE TO A STRAP

- ◆ Pull **Strap** for desired tight fit.



When threading a **Strap**, the **Buckle** piece must have the correct side up. The part the **Strap** goes around has a flat side that belongs on top, and a triangular side that belongs on the bottom.

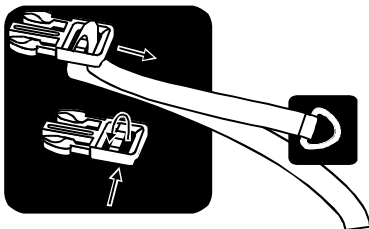


Thread the **Strap** as shown. If the **Strap** has a **Finger Loop**, the **Finger Loop** should be on the top.


HOW TO ATTACH A STRAP TO THE CHAIR

- ◆ Secure **Cam Lock Buckle** and/or **Metal Attaching Clip** to chair, then depending on the width of the strap, thread strap as shown.


Cam Lock Buckle

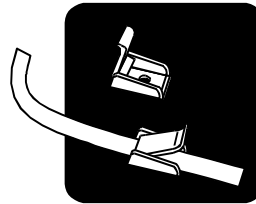


Metal Attaching Clip & 1" Strap

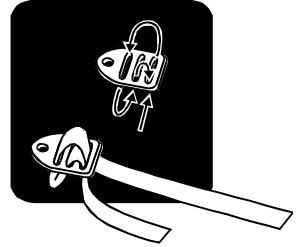


Metal Attaching Clip & 1.5" Strap

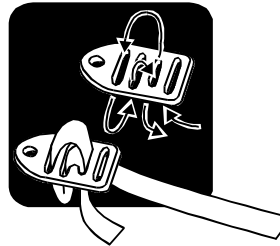

Metal Attaching Clip & 2" Strap

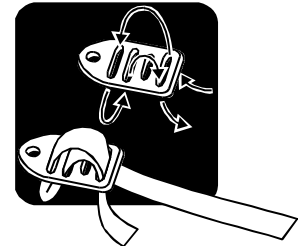

- ◆ Thread the strap through a **3-Bar Slide** as shown, looping strap around the wheelchair tube, and pull tight to hold harness in desired position.

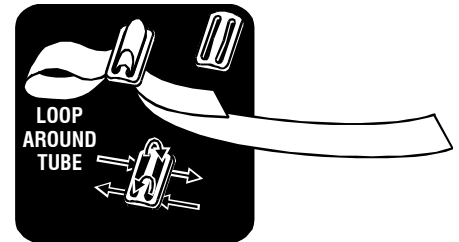

REMOVING USER FROM CHAIR

- ◆ Use **Quick Release Buckles** so you don't have to refit all the adjustments the next time user gets into chair.

IMPORTANT NOTICE

- ◆ **THIS IS NOT A SAFETY DEVICE. IT IS TO BE USED ONLY FOR POSITIONING. IT MUST BE USED IN CONJUNCTION WITH A METAL SEAT BELT!**