# ThevoCalm

DE	Bedienerhandbuch – ThevoCalm.....	3
GB	User manual – ThevoCalm .....	6
FR	Manuel d’utilisation – ThevoCalm.....	9
ES	Manual de usuario – ThevoCalm.....	12
IT	Manuale d’istruzioni – ThevoCalm.....	15
NL	Gebruiksaanwijzing – ThevoCalm .....	19
NO	Bruksanvisning – ThevoCalm.....	21
SE	Bruksanvisning – ThevoCalm.....	24
DK	Brugsanvisning – ThevoCalm.....	27
FI	Käyttäjän käsikirja – ThevoCalm .....	30

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## Dear User,

You have just acquired a product - the ThevoCalm mobility bed - that meets the highest quality standards. ThevoCalm features the latest materials for an extended service life that's guaranteed.

Please read through this manual carefully before using ThevoCalm. It contains lots of useful information and tips.

If you have any further questions or suggestions concerning ThevoCalm give us a call.

Further information is also available from **www.thevo.info**

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## Contents

### Page

Technical data .....	7
Scope of delivery .....	7
Usage .....	7
Disinfection methods and cleaning information .....	7
Application risks and contraindications .....	7
Warranty conditions.....	8

## Technical data

ThevoCalm external dimensions	90 x 200 cm (35.4 x 78.7 ")	100 x 200 cm (39.4 x 78.7 ")	Height
ThevoCalm 80	18.0 kg / 39,7" lbs	20,0 kg / 44,1 lbs	19.0 cm (7.5 ")
ThevoCalm 140	19.5 kg / 43" lbs	21,5 kg / 47.4 lbs	

Other dimensions available on request.

## Scope of delivery

- ThevoCalm with jersey or incontinence cover
- User manual

## Usage

The product is compatible for use with every type of bed. Lay ThevoCalm with the red side facing upwards on the existing bed slats or bed base.

**Important hint:** ThevoCalm can also be used without a pre-existing slatted frame. In this case the supporting surfaces along the sides of the bed base must be at least 13 cm (5.1 ") wide.

**Attention:** Beds with side-rails may require additional side-rail height extenders.

## Disinfection methods and cleaning information

### Mattress

The washable covers are easy to remove thanks to an integrated zip.

### Jersey cover (fabric cover)

- washable at 65 °C (149 °F) • suitable for tumble dryers

### Incontinence cover (rubberised cover)

- washable at 65 °C (149 °F) • suitable for tumble dryers • to be cleaned by wiping disinfection

### Disinfection

We recommend two disinfection methods accepted by the Robert Koch Institute:

- the vacuum/steam/vacuum method up to a maximum temperature of 110 °C (230 °F)
- the steam flow method up to a temperature of 110 °C (230 °F)

**Important hint:** A thermal treatment over 110 °C (230 °F) causes a premature ageing and deformation of the plastic parts and should therefore be avoided. Please make sure when using chemical disinfectants that they are suitable for plastic.

## Application risks and contraindications

ThevoCalm was designed to support movement sequences and optimize the sleep behavior of persons with Parkinson's Disease. ThevoCalm offers a firm surface for lying. Skin condition should be checked regularly for persons with, or at risk for skin breakdown. This particularly applies to immobile patients. Selecting the correct mattress density is extremely important in such cases.

The use of ThevoCalm does not require any special instructions. After practical use and conducting several tests, we are unaware of any problems associated with the use of this product.

When used properly, there are no known risks.

**Important hint:** For the correct supply it is important to ensure that the mattress is selected according to the body weight of the user in question.

- **ThevoCalm 80** up to a body weight of 80 kg (176.4 lbs)
- **ThevoCalm 140** from 100 kg (220.4 lb) up to a body weight of 140 kg (308.6 lbs)

ThevoCalm 140 may also be used in the border zone of patients weighing 80 kg/176.4 lbs (+/- 2 kg / +/- 4.4 lb).

**Caution:** ThevoCalm is not suitable for the prevention and treatment of decubitus ulcers. This requires the use of either ThevoautoActiv or ThevoActiv. We would be pleased to advise you

## Warranty conditions

The company Thomashilfen takes full responsibility for the quality of the product supplied. The highest production standards are used in the selection and processing of materials. All ThevoCalm mattresses are subjected to a thorough final inspection. Thomashilfen offers you a warranty of 24 months. Please retain all relevant ThevoCalm receipts. The warranty covers all claims arising from manufacturing or material defects that affect product function. Damage arising from inappropriate use and natural wear and tear is excluded.

**Important hint:** ThevoCalm is autoclavable and may be re-used. ThevoCalm should be cleaned and disinfected using one of the cleaning methods stated on page 7 with every change of user. Servicing by the manufacturer is not necessary.

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