



# WRIST BRACE WITH THUMB SPICA

## Application Instructions

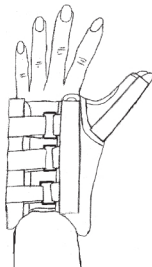

1. Measure the wrist circumference to determine the proper size. Slide brace on the affected hand with the layer of stockinette on the back of the hand.
2. Wrap strap around thumb. Secure hook and loop closure.
3. Thread hook and loop through D-rings securing the hook and loop closures. The metal stay should fit into the palm of the hand and extend down the forearm.
4. Trim the straps leaving approximately 1" (2.54cm) - 2" (5.08cm) of overlap.

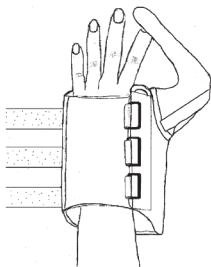

## WASHING INSTRUCTIONS

Remove metal stay. Fasten hook and loop. Hand wash in warm water using a mild detergent. Air dry. Blot excess water with a towel, do not wring out.

10007370