1- and 2-piece open and closed pouches (New Generation)

23311102 v.4 The Coloplast logo is a registered trademark of Coloplast A/S.
© [2011-06.] All rights reserved Coloplast A/S, 3050 Humlebæk, Denmark.

Distributed by Coloplast Corp
Minneapolis, MN 55411
USA

country of origin : Hungary

Manufacturer

Use by

Consult instructions for use

Batch

Warning

Reuse of this single use product may create a potential risk to the user. Reprocessing, cleaning, disinfection and sterilization may compromise product characteristics which in turn create an additional risk of physical harm to or infection of the patient.

Caution

Coloplast accepts no liability for any injury or other loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

Please seek advice from your stoma care nurse or doctor before using a convex product.

Intended use

The Assura/Alterna ostomy adhesive skin barrier and ostomy pouch are intended to collect output from a stoma. The adhesive skin barrier is intended to adhere to peristomal skin and is used for positioning of ostomy appliances.

Information

Filter

Most of the pouches have an integrated filter. Each box contains filter covers which can be used when showering, swimming or to prevent gas from escaping from the bag. The filter cover is placed on the opening of the filter.

Soft Cover


After showering, pat your bag dry with a towel.

Two different outlets on open bags

The Assura/Alterna drainable pouch range is available with both an EasiClose™ outlet and a clamp closure system.

The range with the EasiClose™ outlet has an integrated closure system, eliminating the need for clamps.

Illustration of the EasiClose™ outlet – front and back

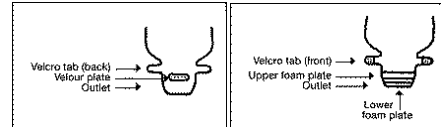How to use

1

2

3

4

5

1-piece

2-piece

6

1-piece

2-piece

7

1-piece

2-piece

8

9


10

11

12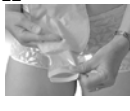

Closing of the pouch before use

Pouches with EasiClose™ outlet:

1. Place the bag on the table, adhesive side down. Fold the lower small foam plate up and over the upper large foam plate.
2. Fold the outlet over the upper large foam plate, until the velour sheet and the velcro® tabs are situated opposite each other.
3. Seal the outlet by folding the velcro® tabs onto the velour plate and press firmly. To hide the outlet, you may fold once more under the soft cover, as described in step 14.

Pouches with clamp closure

4. Remove a clip from the sheet and remove the protective paper. Place the adhesive side down at the opening. If the clip is not placed correctly the first time, remove and reposition it. Fold up to 4-5 times working towards the skin and then fold back the ends of the clip. With a hard clamp it is only necessary to fold the bag once before closing the clamp.

5. Cutting of the skin barrier

Trace the size and shape of your stoma onto the stoma guide. Cut the hole in the skin barrier to fit the size and shape of your stoma. Cutting is easier if you use a curved pair of scissors. Do not cut beyond the last circle on the skin barrier. Ensure that your skin is clean and dry before you proceed with the application of the pouch.

6. Removal of the protective backing

Remove the transparent protective backing. Place one thumb on the tab and pull the protective backing away with the other hand.

7. Application of the adhesive

Place the adhesive on the skin ensuring a snug fit around the stoma. Press firmly against the skin starting at the bottom and smoothing upwards with your fingers.

2-piece system

8. Application of the pouch

Ensure the lock ring is open. Press pouch and skin barrier together starting at the bottom. Run your fingers around the coupling on the outside of the bag from the bottom to the top. When a slight click is heard, the pouch is securely positioned on the skin barrier. The pouch may then be turned to its optimal position.

Locking of the coupling

Then close the lock ring. When you hear a click, the pouch is securely locked to the skin barrier.

9. Removal of the pouch and skin barrier

The pouch:

1. Open the lock ring by pressing the small tab with the tip of your finger.

2. Remove the pouch from the skin barrier by lifting up and pulling away from the skin barrier at the same time. This is easier if you hold onto the top of the pouch.

The skin barrier:

The skin barrier is removed from the skin by peeling downward slowly and carefully at the same time holding the skin taut with the other hand.

1-piece system

10. Removal of pouch

The pouch is removed by gently pulling the tab and removing it from the skin. Keep the skin taut with the other hand.






11. Opening of the pouch

Take hold of the end of the velcro® tab. Lift both velcro® tabs off the velour plate. It is advised to unfold the outlet where you want to empty the pouch.

For pouches with clip closure: Unfold the ends and roll down the outlet and empty the pouch.

12. Emptying of the pouch

Allow the contents to run out while you control the emptying with your fingers. Ensure that the pouch is completely emptied.

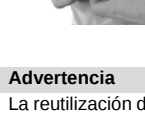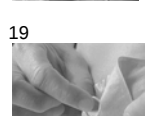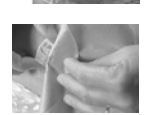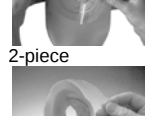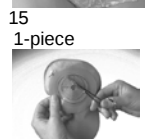13. Cleaning of the outlet

Use a tissue or dry wipe to clean the outlet in one sliding movement away from your body. Fold the small plate backwards so only the large plate is visible (EasiClose™ outlet only). The outlet is now clean and ready to be closed again.

Closing of the outlet

Follow step 1-4

14. Cosmetic closing of the pouch (EasiClose™ pouches only)

The closing system can be folded under the soft fronting. This will make the pouch more discreet.

Assura/Alterna® 1- and 2-piece closed pouches

15. Cutting of the skin barrier

Trace the size and shape of your stoma onto the transparent protective backing.

Cut the hole in the skin barrier to fit the size and shape of your stoma. Cutting is easier if you use a curved pair of scissors. Do not cut beyond the last circle on the skin barrier.

Ensure that your skin is clean and dry before you proceed with the application of the pouch.

16. Removal of the protective backing

Remove the transparent protective backing. Place one thumb on the tab and pull the protective backing away with the other hand.

17. Application of the adhesive

Place the adhesive on the skin ensuring a snug fit around the stoma. Press firmly against the skin starting at the bottom and smoothing upwards with your fingers.

2-piece system

18. Application of the pouch

Ensure the lock ring is open. Press pouch and base plate together starting at the bottom. Run your fingers around the coupling on the outside of the pouch from the bottom to the top. When a slight click is heard the pouch is securely positioned on the base plate. The pouch may then be turned to its optimal position.

Locking of the pouch

Then close the lock ring. When you hear a click, the pouch is securely locked to the skin barrier.

19. Removal of the pouch and skin barrier

The bag:

1. Open the lock ring by pressing the small tab with the tip of your finger.
2. Remove the pouch from the base plate by lifting up and pulling away from the skin barrier at the same time. This is easier if you hold onto the top of the pouch.

The base plate:

The skin barrier is removed from the skin by peeling downward slowly and carefully at the same time holding the skin taut with the other hand.

1-piece system

20. Removal of pouch

The pouch is removed by gently pulling the tab and removing it from the skin. Keep the skin taut with the other hand.

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