 Coloplast A/S
3050 Denmark
23310843 The Coloplast logo is a registered trademark of Coloplast A/S.
© [2011-02.] All rights reserved Coloplast A/S, 3050 Humlebæk, Denmark.

Warning

Reuse of this single use product may create a potential risk to the user. Reprocessing, cleaning, disinfection and sterilization may compromise product characteristics which in turn create an additional risk of physical harm to or infection of the patient.

Caution

Please seek advice from your stoma care nurse or doctor before using Convex Light products.

Coloplast accepts no liability for any injury or other loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

Introduction

The SenSura ostomy adhesive barrier and pouch are intended to collect output from a stoma. The SenSura adhesive barrier is intended to adhere to intact peristomal skin and is used for positioning of ostomy pouches.

Adhesive

The adhesive is designed to provide secure fit and adhesion to individual body contours. It protects the skin and follows body movements without compromising adhesion. It keeps the skin underneath the adhesive healthy by absorbing excess moisture.

Filter and filter label

The filter(s) allows air to escape the pouch and removes unpleasant odour for more discretion. If you want to prevent air escaping the pouch, simply stick a cover onto the opening of the filter. Each box is provided with a set of filter covers.

Soft cover

The soft cover is made of strengthened fabric and is water repellant. The inspection window, available on some product versions, helps to apply the adhesive securely around the stoma.

For Convex Light/Soft Seal only

Convex Light products are designed for those people who have an ostomy that is flush (skin level), retracted (below skin level) or ostomies located in skin folds. The appliance comes with a belt attachment on either side.

How to use open and closed pouches

1. Cutting

Trace the size and shape of the stoma on the stoma guide. Cut the hole to fit the size and shape of the stoma. To reduce the risk of leakage and skin problems it is very important that the hole in the adhesive fits perfectly around the stoma. Cutting is easier if you use a curved pair of scissors. Ensure that the skin is clean and dry before you proceed with the application of your pouch. If you have an open pouch please remember to close the outlet – see point 5

2. Removal of protective film

Place your thumb on the white tab and pull the blue/turquoise release tab away to remove the protective film.

3. Application

Apply the adhesive around the stoma. To ensure perfect fit, fold the pouch backwards, align the adhesive around the stoma and press it firmly onto the skin. Start at the bottom and smooth it upwards with your fingers.

4. Removal

Pull down the white removal tab to loosen the adhesive from the skin. Gently remove the adhesive by rolling it step by step downwards. Apply light pressure on your skin with your other hand.

How to use the Hide-away/EasiClose outlet on open pouches

Close the outlet before use

5. Place the pouch on a table or a flat area so the adhesive is downwards. Bend the lower, small plate over the upper, large plate.

6. Fold the outlet twice – until you see the velour plate.

7. Seal the outlet by folding the hook (Velcro) ears onto the velour plate and press firmly.

For some product versions you can, if desired, fold once more as described under "Cosmetic closing of the Outlet" (point 13). These product versions aren't available in all countries.

8. Opening of pouch

Take hold of the end of the hook (Velcro) ears. Lift both hook (Velcro) ears off the velour plate. Unfold the outlet in the position where you want to empty the pouch.

9. Emptying of the pouch

Allow the contents to run out while you control the emptying with your fingers.

10. Cleaning of the outlet

Use a tissue or dry wipe to clean the outlet in one sliding movement away from your body.

11. Fold the small plate backwards so that only the large plate is visible. You can now clean the edge. The outlet is now cleaned and ready to be closed again.

12. Re-closing of the outlet

Bend the lower small plate over the upper large plate. Then fold the outlet twice – until you see the velour plate. Seal the outlet by folding the hook (Velcro) ears onto the velour plate and press firmly.

13. Cosmetic closing of the outlet (only for some product versions)

The whole closing system can be folded up under the soft cover and attached to the Velcro dot. The pouch will now become more discreet.

Distributed by Coloplast Corp
Minneapolis, MN 55411
USA

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Distribuido por Coloplast Corp