ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INDICATIONS: Acute ankle sprains, chronic ankle sprains, prophylactic use.

CONTRAINDICATIONS: None

WARNINGS AND PRECAUTIONS: If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately. Never wear the support over an open wound. Ankle brace should be worn with an athletic sock.

APPLICATION INFORMATION:

A) **Stabilizing PRO Ankle** – Before application, loosen straps and laces. The ankle must be at a 90° angle when applying the brace. While seated at the edge of a chair, place foot in ankle brace. Make sure the tongue is on the top of the foot area prior to lacing-up. Lace up the brace like a shoe.

B) **Stabilizing Speed PRO Ankle** - Before application, loosen straps and laces. The ankle must be at a 90° angle when applying the brace. While seated at the edge of a chair, place foot in ankle brace. Make sure the tongue of brace is on the top of the foot area prior to lacing-up. Pull lace tabs across top of foot and secure tab to Velcro on the opposite side of the brace.

C) For application to the right foot, slightly turn heel out. For the left foot, slightly turn heel in. Starting with the long **outer** strap, wrap in a figure 8 over forefoot and then under the arch area of the foot. Pull strap vertically until firm tension is achieved. Secure to Velcro. Trim strap if necessary.

D) For application to the right foot, slightly turn heel out. For the left foot, slightly turn heel in. Starting with the long **inner** strap, wrap in a figure 8 over forefoot and then under the arch area of the foot. Pull strap vertically until firm tension is achieved. Secure to Velcro. Trim strap if necessary.

E) Wrap the short circumferential straps around both sides of the ankle over the long inner and outer straps. Secure at the front of the brace.

Note: Ankle brace should be snug but not impair circulation.

CLEANING INSTRUCTIONS:

Hand wash with mild soap in lukewarm water. Rinse thoroughly. AIR DRY.

Note: If not rinsed thoroughly, residual soap may cause skin irritation or deteriorate the material.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

FOR SINGLE PATIENT USE ONLY.

LATEX FREE.

MATERIAL CONTENT:

Polyester; Nylon; Polypropylene; Plastic

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