


## Connecting the air hose and hand control

- Connect the air hose from the Bathing Cushion to the Airflo 12 air outlet socket (**D** fig. 4).
- Connect the Hand Control (**E**) to the Airflo 12 hand control socket (**F**).

## Dry trials (without water in the bath)

- Select 'Up' on the hand control (**G** fig. 5) to inflate the Bathing Cushion. The compressor will stop automatically once the Bathing Cushion is fully inflated. Press firmly down on the Bathing Cushion to secure the suction cups to the floor of the bath. Check all four suction cups are securely stuck to the bath. Wet them to help them stick if necessary.
- Whilst still clothed and with the help of an assistant if required, carry out some dry 'practice runs' according to these instructions until you are totally confident using the product.

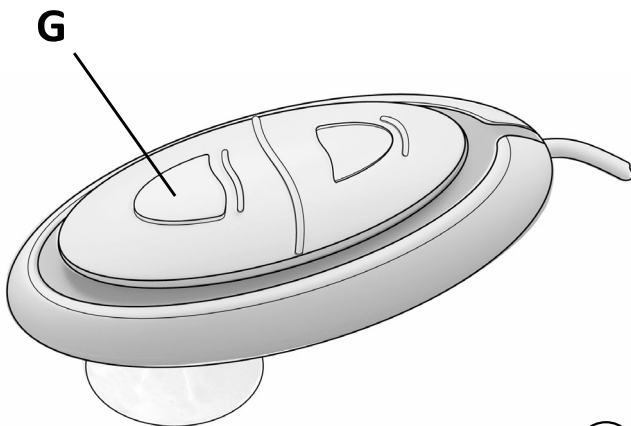

4

5

**Note:** During a 'dry trial' the Bathing Cushion may make a noise when being lowered and raised due to the fabric rubbing against the dry sides of the bath, this is normal and the noise will disappear when the bath is full of water.