## ENGLISH

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

### INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient or the patient's caregiver.

The user or caregiver should be able to read, understand and be physically capable of performing all the directions, warnings and cautions provided in the information for use.

**INTENDED USE/INDICATIONS:** The Procure Industrial Back Support is designed to provide comfort and support of the lumbar spine. It may be suitable for the symptomatic relief of acute and chronic low back pain.

Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

**CONTRAINDICATIONS:** If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.

### WARNINGS AND PRECAUTIONS:

- For maximum efficacy, ensure the correct size of strap is selected and the side pulls are adjusted to the required fit.
- If you experience any pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your medical professional immediately.
- Do not use over open wounds.
- Do not use this device if it was damaged and/or package has been opened.

**NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

### APPLICATION INFORMATION:

1. Center support on lower back. Pull the waistbands over the hips and secure in front forming a "V" below the navel. This is neutral position. (Fig. 1)
2. When performing physical tasks in supports with double side pulls, grasp outer straps with both hands, stretch forward and secure. Upon completion of task, release outer straps and return to neutral position. (Fig. 2)

For optional shoulder strap use, attach to support with suspender buckles in front.

Adjust suspenders for comfort and fit.

**CAUTION:** Support should be worn as directed, never wear unfastened.

**CLEANING INSTRUCTIONS:** Hand wash in cold water with mild detergent. Air dry. Avoid direct heat or sun.

**WARRANTY:** DJO, LLC will repair or replace all or part of unit and its accessories for material and workmanship defect for a period of six months from the date of sale.

### FOR SINGLE PATIENT USE ONLY.

**NOT MADE WITH NATURAL RUBBER LATEX.**

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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