

THE LEADERS IN RECREATION, FITNESS AND ABILITY!

9889 Garrymore Ln Missoula, MT 59808 888-687-3552 | +1-406-549-0769 www.aquacreek.com

## Sling Assembly Instructions

PART #: F-036RSS

**MANDATORY** LEAVE THIS MANUAL WITH LIFT OWNER

Read and follow all instructions. Lift safety can only be ensured if the lift is installed and operated according to these instructions.


- NEVER permit children to play on or around the lift
- Do not allow children to use the lift without adult supervision
- NEVER apply direct water pressure to the electronics
- NEVER use the lift with a dry pool

## **READ CAREFULLY**

Check entire box and all packing materials for parts. Before beginning assembly, read the instructions and identify parts using the figures and parts listed in this document.

It is critical that all parts be carefully inspected prior to installation. If any damage occurred in transit, Aqua Creek Products, LLC must be notified within three days of receipt of unit.

Proper installation cannot be overstressed, as an improper installation voids Aqua Creek's warranty and may affect the safety of the user.

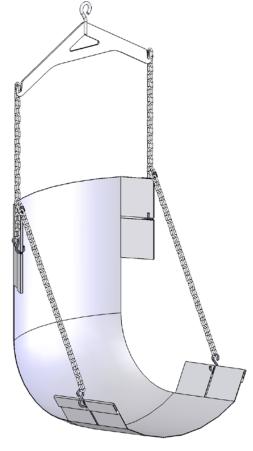

## **Sling Assembly**

To assemble the sling so that it is ready for use, simply follow the outline below:

-Slide the four (4) sling bars into the four corner pockets of the sling.

-Run the seatbelt through the three center loops and connect together on the opposite side.

-Slide the stiffener plate into the large pocket towards the front of the sling.

-Attach the sling to the chain set and hanger bar.

Your sling is now assembled and ready for use!

