








Instructions for Use

---

FORMFIT<sup>®</sup> POST-OP KNEE 16"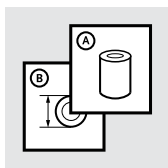

		3
EN	Instructions for Use	4
DE	Gebrauchsanweisung	6
FR	Notice d'utilisation	9
ES	Instrucciones para el uso	12
IT	Istruzioni per l'uso	15
NO	Bruksanvisning	18
DA	Brugsanvisning	20
SV	Bruksanvisning	22
EL	Οδηγίες Χρήσης	24
FI	Käyttöohjeet	27
NL	Gebruiksaanwijzing	29
PT	Instruções de Utilização	32
PL	Instrukcja użytkownika	35
CS	Návod k použití	38
TR	Kullanım Talimatları	40
RU	Инструкция по использованию	42
JA	取扱説明書	45
ZH	中文说明书	47
KO	사용 설명서	49
SK	Návod na použitie	52
ET	Kasutusjuhend	54
RO	Indicații pentru Utilizare	57
AR	تعليمات الاستخدام	60
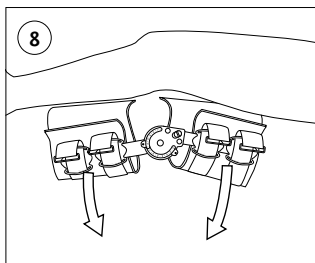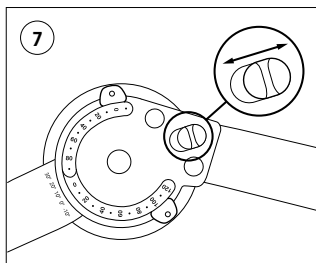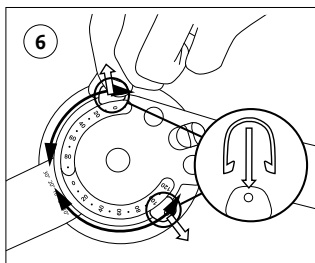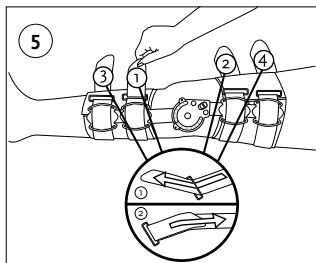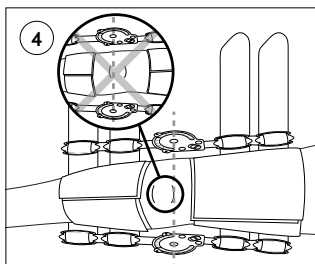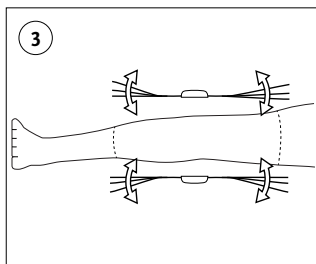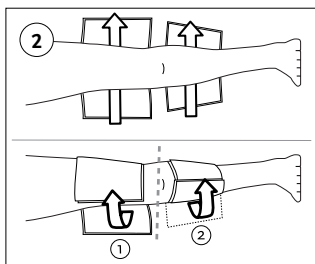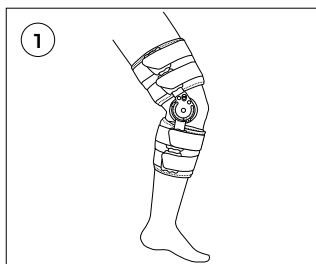
# ENGLISH

---

## DESCRIPTION OF SYMBOLS

**MD** Medical Device

## INDICATIONS FOR USE

Post-operative/post-injury ROM control or immobilization.  
Ligament and fracture-related surgeries and when controlled range of motion is necessary.

## CONTRAINDICATIONS

None known.

## INTENDED USE

The Formfit® Post-Op Knee 16" is intended to be used for knee immobilization and range of motion restriction (**Figure 1**).

## WARNINGS AND PRECAUTIONS

**WARNING:** Do not use for multiple patients. Only use the device for a single patient.

**CAUTION:** If you experience increased pain, swelling, sensation changes or any adverse reactions while using this product, immediately consult your medical professional.

**Fitting by a professional required.**

## FITTING INSTRUCTIONS

### DONNING THE BRACE

**WARNING:** Make sure to keep the leg straight when you slide the foam pads and straps under it.

1. Unfasten the leg straps.
2. Fully pull the open ends of the straps through the double D-rings to remove them from the left strut.
3. Remove the foam pads from the two struts.
4. Fully open the foam pads.
5. Wrap the foam calf and thigh pads tightly around the leg and fasten at the front (**Figure 2**).
6. Put the struts on each side of the leg. If necessary, use bending tools to adapt the brace to the patient's leg shape (**Figure 3**).
7. Attach the struts to the foam pads on each side of the leg. Align the middle of the hinge joint with the top of the patella and the mid-line of the knee (**Figure 4**).
8. Carefully slide the straps under the foam wraps.
9. Put the straps back through the double D-rings.
10. Pull and fasten the straps below the knee first; followed by the strap above the knee. Complete the remaining straps alternating below and above the knee until all straps are complete (**Figure 5**).

**CAUTION:** Do not tighten the straps too much. If the straps are too tight, the blood circulation can be cut off.

### Setting Range of Motion

Adjust the ROM as follows (**Figure 6**):

Extension: Pull the black extension button up and set the angle.

Extension can be limited at: -10°, 0°, 10°, 20°, 30°, 40°, 50°, 60°, 70°, 80°, 90°.

Flexion: Pull the white flexion button up and set the angle. Flexion can be limited at: 0°, 10°, 20°, 30°, 40°, 50°, 60°, 70°, 80°, 90°, 100°, 110°, 120°.

Release a button to let it click into place. Put the stop locks onto the buttons to secure them **(Figure 6)**.

### **Immobilizing the Knee**

Use the drop lock to fully immobilize the knee. It can be set at -10°, 0°, 10°, 20°, 30° of flexion. An arrow shows the angle. The angle is shown by the scale on the strut, below the hinge.

### **Adjust immobilization as follows (Figure 7):**

1. Push the blue drop-lock button up to unlock the hinge.
2. Move the brace until the arrow and the correct angle agree.
3. Push the blue drop-lock button down to lock the hinge angle.

### **Doffing the Brace**

1. Unfasten the straps and the foam pads.
2. Remove the brace **(Figure 8)**. You can put it back on again fully assembled.

## **CLEANING AND CARE**

The foam pads can be hand washed with mild soap. Leave to dry. Do not tumble dry.

## **SUPPLEMENTARY PRODUCTS**

### **Replacement Parts**

- Replacement Softgoods
- ROM Stop Locks

## **REPORT A SERIOUS INCIDENT**

### **Important notice to users and/or patients established in Europe:**

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## **DISPOSAL**

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

## **LIABILITY**

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

## **WARRANTY**

Plastic parts and hinges 3 months warranty. Foam pads and leg straps 30 days. 6 months free from defects.

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