










Instructions for Use








FORMFIT[®] POST-OP KNEE

24" & 28"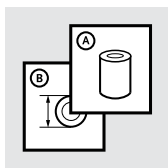

		3
EN	Instructions for Use	5
DE	Gebrauchsanweisung	8
FR	Notice d'utilisation	11
ES	Instrucciones para el uso	14
IT	Istruzioni per l'uso	17
NO	Bruksanvisning	20
DA	Brugsanvisning	23
SV	Bruksanvisning	26
EL	Οδηγίες Χρήσης	29
FI	Käyttöohjeet	32
NL	Gebruiksaanwijzing	35
PT	Instruções de Utilização	38
PL	Instrukcja użytkownika	41
CS	Návod k použití	44
TR	Kullanım Talimatları	47
RU	Инструкция по использованию	50
JA	取扱説明書	53
ZH	中文说明书	56
KO	사용 설명서	59
SK	Návod na použitie	62
ET	Kasutusjuhend	65
RO	Indicații pentru Utilizare	68
AR	تعليمات الاستخدام	71

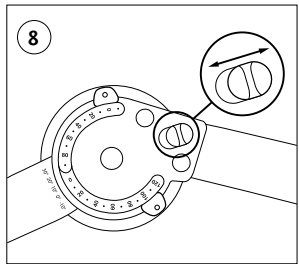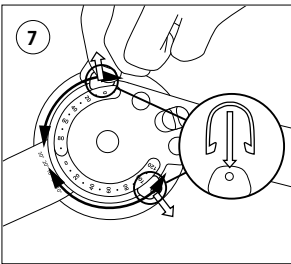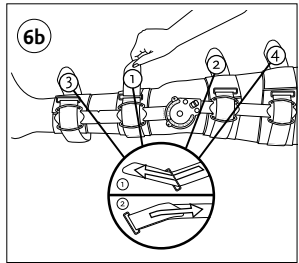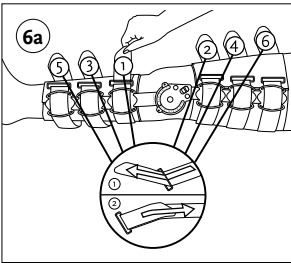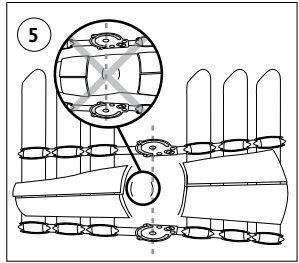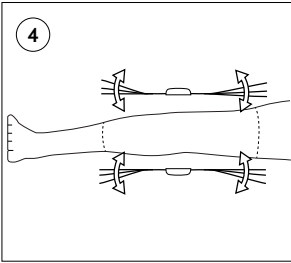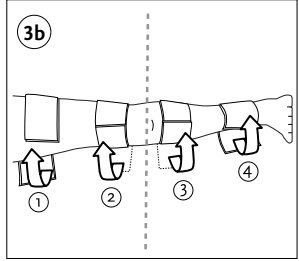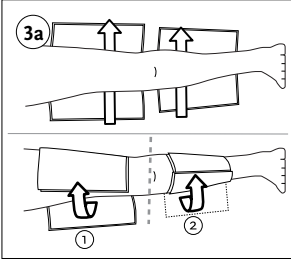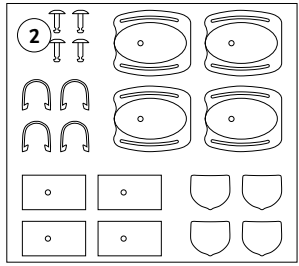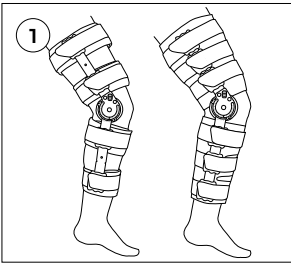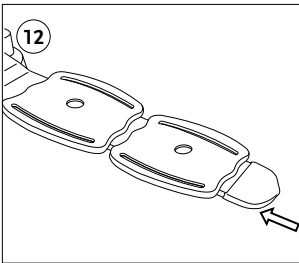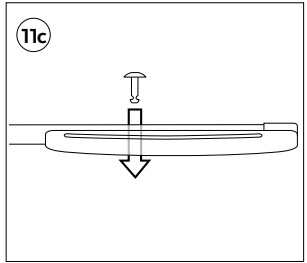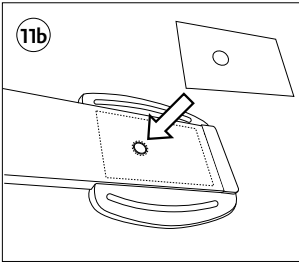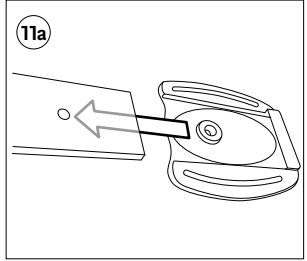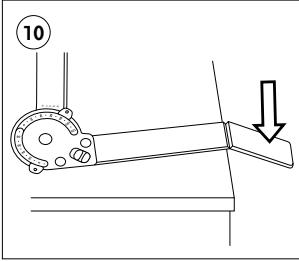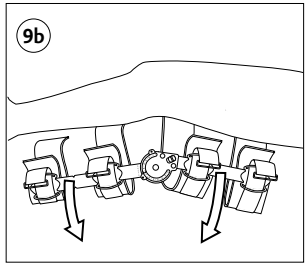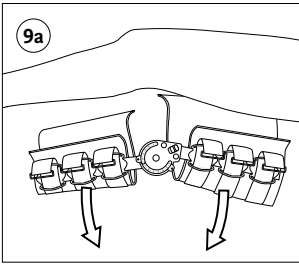ENGLISH

Medical Device

INDICATIONS FOR USE

Post-operative/post-injury ROM control or immobilization. Ligament and fracture-related surgeries and when controlled range of motion is necessary.

CONTRAINDICATIONS

None known.

INTENDED USE

The Formfit® Post-Op Knee 24" and 28" is intended to be used for knee immobilization and range of motion restriction. The brace comes in two versions: Cool Foam and Full Foam (**Figure 1**).

WARNINGS AND PRECAUTIONS

WARNING: Do not use for multiple patients. Only use the device for a single patient.

CAUTION: If you experience increased pain, swelling, sensation changes or any adverse reactions while using this product, immediately consult your medical professional.

Fitting by a professional required.

FITTING INSTRUCTIONS

DONNING THE BRACE

WARNING: Make sure to keep the leg straight when you slide the foam pads and straps under it.

1. Unfasten the leg straps.
2. Fully pull the open ends of the straps through the double D-rings to remove them from the left strut.
3. Remove the foam pads from the two struts.
4. Fully open the foam pads.
5. Wrap the foam calf and thigh pads tightly around the leg and fasten at the front.
Full Foam: (**Figure 3a**). Cool Foam: (**Figure 3b**).
6. Put the struts on each side of the leg. If necessary, use bending tools to adapt the brace to the patient's leg shape (**Figure 4**).
7. Attach the struts to the foam pads on each side of the leg. Align the middle of the hinge joint with the top of the patella and the mid-line of the knee (**Figure 5**).
8. Carefully slide the straps under the foam wraps.
9. Put the straps back through the double D-rings.
10. Pull and fasten the straps below the knee first; followed by the strap above the knee. Complete the remaining straps alternating below and above the knee until all straps are complete.
Full Foam: (**Figure 6a**). Cool Foam: (**Figure 6b**).

CAUTION: Do not tighten the straps too much. If the straps are too tight, the blood circulation can be cut off.

Setting Range of Motion

Adjust the ROM as follows (**Figure 7**):

Extension: Pull the black extension button up and set the angle. Extension can be limited at: -10°, 0°, 10°, 20°, 30°, 40°, 50°, 60°, 70°, 80°, 90°.

Flexion: Pull the white flexion button up and set the angle. Flexion can be limited at: 0°, 10°, 20°, 30°, 40°, 50°, 60°, 70°, 80°, 90°, 100°, 110°, 120°.

Release a button to let it click into place. Put the stop locks onto the buttons to secure them (**Figure 7**).

Immobilizing the Knee

Use the drop lock to fully immobilize the knee. It can be set at -10°, 0°, 10°, 20°, 30° of flexion. An arrow shows the angle. The angle is shown by the scale on the strut, below the hinge.

Adjust immobilization as follows (**Figure 8**):

1. Push the blue drop-lock button up to unlock the hinge.
2. Move the brace until the arrow and the correct angle agree.
3. Push the blue drop-lock button down to lock the hinge angle.

Doffing the Brace

1. Unfasten the straps and the foam pads.
2. Remove the brace (**Figure 9a, 9b**). You can put it back on again fully assembled.

SHORTENING THE BRACE

If necessary, the 24" and 28" braces can be shortened to 16".

A shortening kit is included for Cool Foam versions (**Figure 2**). **Note:** Plastic End Caps are not part of the Cool Foam shortening kit; they are only part of the Full Foam shortening kit.

WARNING: Only shorten the brace if ordered by the treating doctor. Shortening the brace reduces the function of the brace.

Shorten the struts as follows:

1. Remove the brace from the patient.
2. Fully remove the straps and foam pads from the struts.
3. Put a strut on a table.
4. Align a seam with the edge of the table.
5. Hold the strut tight and break off at the seam (**Figure 10**).
6. Repeat for the other end.

Cool Foam

1. Put the double D-rings from the kit onto each shortened end of the struts. Make sure the double D-rings are on the outer side (**Figure 11a**).
2. Attach the hook and loop fastener onto the strut over the rivet hole (**Figure 11b**).
3. Put the plastic rivet through the hole and the double D-ring from the inside of the strut (**Figure 11c**).
4. Assemble the brace again (refer to "Donning the Brace" above).

Full Foam

1. Make sure the edges are smooth and apply the plastic end caps (**Figure 12**). If necessary, use a file to round the edges of the strut.
2. If necessary, use scissors to trim the foam pads.
3. Assemble the brace again (refer to "Donning the Brace" above).

CLEANING AND CARE

The foam pads can be hand washed with mild soap. Leave to dry. Do not tumble dry.

SUPPLEMENTARY PRODUCTS

Replacement Parts

- Replacement Softgoods
- ROM Stop Locks

Accessories

- Strap Reduction Kit (Full foam)
- Strap Reduction Kit (Cool foam)

REPORT A SERIOUS INCIDENT

Important notice to users and/or patients established in Europe:

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

WARRANTY

Plastic parts and hinges 3 months warranty. Foam pads and leg straps 30 days. 6 months free from defects.

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