



Application of Grip-Lok® 3300M Securement Device

Prepare the skin according to the standard hospital protocol for dressing application. Skin must be **CLEAN** and **DRY** prior to Grip-Lok application. For full instructions, please consult the IFU at tidiprotects.com/grip-lok.

STEP
1



Open the top fabric over-strap and remove inner release liner to expose adhesive.

STEP
2



Position the Grip-Lok under the line/hub. Place the line/hub in the center of the bottom adhesive strip.

STEP
3



Secure the top strap over the line/hub and press the adhesive in place. Pull and remove one side of the bottom release liner and secure the Grip-Lok to the skin. Then, remove the other side of the bottom release liner and secure the Grip-Lok to the skin.

STEP
4



The top strap can be peeled back to inspect and adjust the line/hub.

