

USER MANUAL

---

# OAKWORKS® Treatment Tables




Powerline Series



Seychelle™ Series

made in the USA  
with US & imported parts

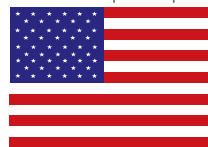

[www.oakworks.com](http://www.oakworks.com) · 717.235.6807

**OAKWORKS®**

Printed in USA

All rights are reserved. No part of this document may be photocopied, reproduced or translated to another language without prior written consent of OAKWORKS®, Inc.

OAKWORKS® is a registered trademark of OAKWORKS®, Inc.

#### Notice

The information contained in this document is subject to change without notice and should not be construed as a commitment by OAKWORKS®, Inc.

OAKWORKS®, Inc. assumes no responsibility for any errors that may appear in this document nor does it make expressed or implied warranty of any kind with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

OAKWORKS®, Inc. shall not be liable for incidental or consequential damages in connection with or arising out of the furnishing, performance, or use of this document and the program material which it describes.

# TABLE OF CONTENTS

---

- Product Use Description** ..... 1
- Important Safety Instructions** ..... 1
- Product Description & Photo**
  - Powerline Series .....2-3
  - Seychelle Series.....4-5
- Installation**
  - Table Assembly.....6
  - Inset Shelf.....6
- Directions for Use**
  - Paper Roll Holder Use ..... 7
  - Wave Backrest Use ..... 7
  - Table Extender ..... 7
  - QuickLock Face Rest ..... 8
- Cleaning & Disinfection** .....9
- Inspections & Maintenance** .....9
- Warranty Information** .....9
- Model Number & Serial Number** ..... 10
- Specifications**
  - Product Specifications ..... 11
  - Environmental Conditions ..... 11
- Contact Information** ..... back cover

---

## PRODUCT USE DESCRIPTION / IMPORTANT SAFETY INSTRUCTIONS

---

### PRODUCT USE DESCRIPTION

The Oakworks Powerline & Seychelle™ Tables are used to support the patient during examinations and procedures. They are intended to be operated by a professional in an exam/treatment environment. No special training is required but a review of the following manual is important for the safety of the operator and patient. The professional should read and understand this entire manual before use with a patient. There are no known contradictions to the use of this equipment.

### IMPORTANT SAFETY INSTRUCTIONS



#### CAUTION

#### READ AND SAVE THESE INSTRUCTIONS



This symbol, when used in this manual and on product labels, represents a caution warning. Be sure to read and comply with all precautions and warnings.

The Backrest section is not designed to support the entire weight of the patient. Do not sit on the Backrest section.

The table is designed to be a stand alone table. This table must not be modified or incorporated into any other equipment.

There are potential pinch points around and underneath the tabletop. It is the responsibility of the operator of this equipment to insure that bystanders are not in the area below or around this equipment during operation.

Proper operation of this equipment is very important for the safety of the operator, patient, and any other individuals in the area of this equipment. Directions for use of this equipment are described in this manual. The operator should read these sections carefully.

Do not use outdoors.

Use this furnishing only for its intended use as described in this manual. Do not use attachments not recommended by Oakworks.

**Weight Limit: (patient and accessories) 500 lbs./227 kg. Do not exceed.**

## PRODUCT DESCRIPTION

### *Powerline Flat Top*



STANDARD SPECIFICATIONS	
Height Range	Fixed 30" (76 cm.)
Table Capacity	500 lbs. (226 kg.) (patient and accessories)
Widths	27" (69 cm.) or 30" (76 cm.)
Length	72" (183 cm.)
Padding	2.5" (6 cm.) Semi-Firm
Upholstery	Terra Touch™
Paper Roll Holder	Accommodates up to 21" (53 cm.) rolls (roll not included)
Table Weight*	90 lbs. (41 kg.)
Warranty	3 year parts, 1 year labor
Safety Listings	CE marked, FDA
OPTIONS & ACCESORIES	
QuickLock Face Rest	
Table Extender	
Inset Shelf (adds 25 lbs. (11 kg.))	
Padding - 1.75" (4 cm.) Firm Response	

# PRODUCT DESCRIPTION

## *Powerline Wave Backrest*



QuickLock Face Rest (accessory)


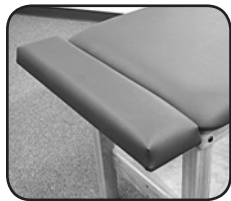


Table Extender (accessory)




Wave Backrest

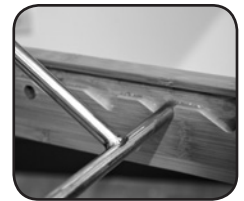

STANDARD SPECIFICATIONS	
Table Height	Fixed 30" (76 cm.)
Wave Backrest	Manual 0-60°, 6 set positions
Table Capacity	500 lbs. (226 kg.) (patient and accessories)
Widths	27" (69 cm.) or 30" (76 cm.)
Length	72" (183 cm.)
Padding	2.5" (6 cm.) Semi-Firm
Upholstery	Terra Touch™
Paper Roll Holder	Accommodates up to 21" (53 cm.) rolls (roll not included)
Table Weight*	90 lbs. (41 kg.)
Warranty	3 year parts, 1 year labor
Safety Listings	CE marked, FDA
OPTIONS & ACCESORIES	
QuickLock Face Rest	
Table Extender	
Inset Shelf (adds 25 lbs. (11 kg.))	
Padding - 1.75" (4 cm.) Firm Response	

## PRODUCT DESCRIPTION

### *Seychelle Flat Top*

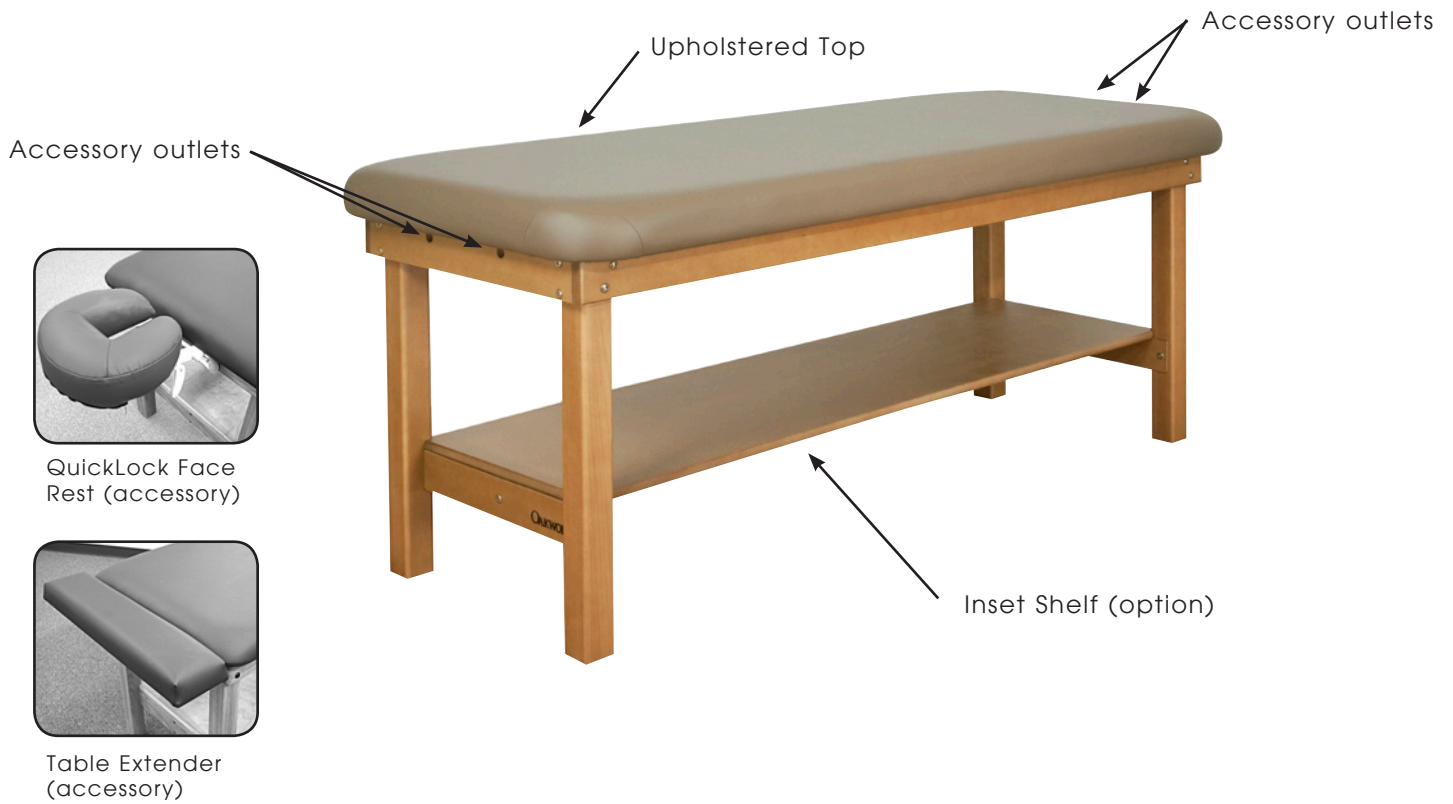

STANDARD SPECIFICATIONS	
Height Range	Fixed 30" (76 cm.)
Table Capacity	500 lbs. (226 kg.) (patient and accessories)
Widths	27" (69 cm.) or 30" (76 cm.)
Length	72" (183 cm.)
Padding	3" (8 cm.) Comfort
Upholstery	Terra Touch™
Table Weight*	90 lbs. (41 kg.)
Warranty	3 year parts, 1 year labor
Safety Listings	CE marked
OPTIONS & ACCESORIES	
QuickLock Face Rest	
Table Extender	
Inset Shelf (adds 25 lbs. (11 kg.))	
Padding - 4" (10 cm.) Comfort	

## PRODUCT DESCRIPTION

### *Seychelle Wave Backrest*



QuickLock Face Rest (accessory)



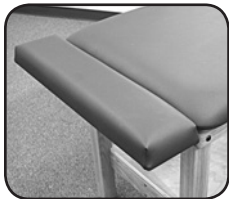


Table Extender (accessory)

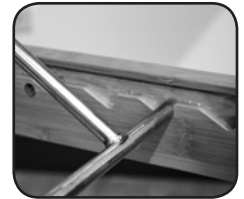

Wave Backrest

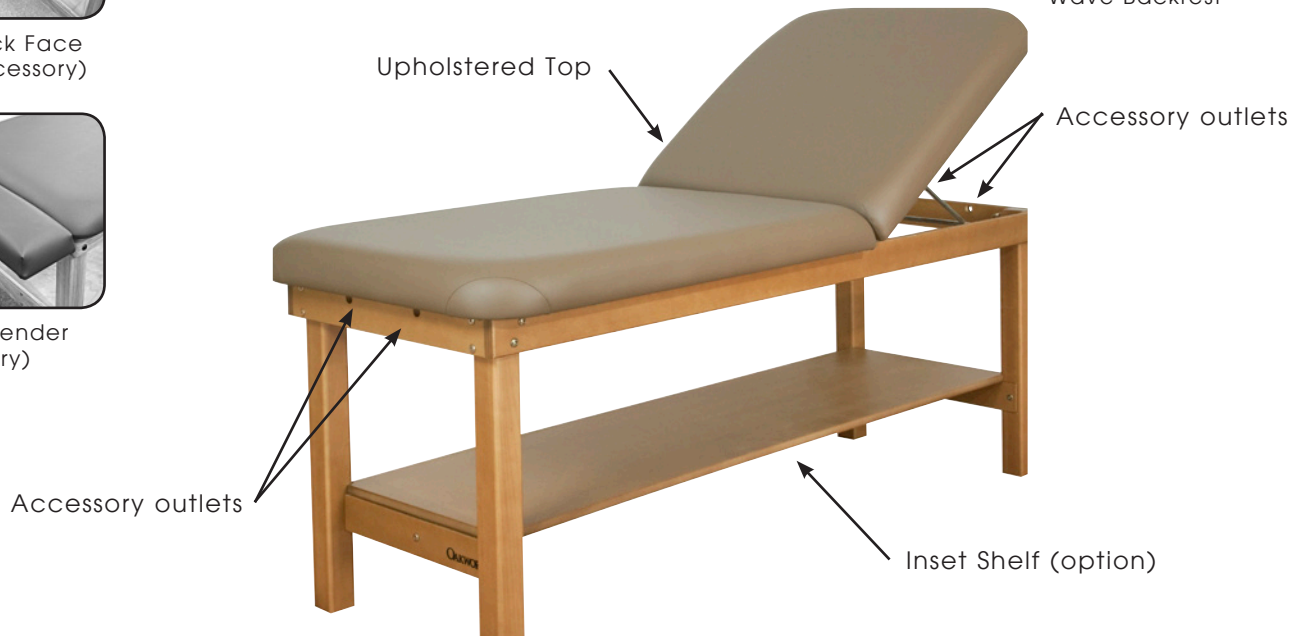

STANDARD SPECIFICATIONS	
Table Height	Fixed 30" (76 cm.)
Wave Backrest	Manual 0-60°, 6 set positions
Table Capacity	500 lbs. (226 kg.) (patient and accessories)
Widths	27" (69 cm.) or 30" (76 cm.)
Length	72" (183 cm.)
Padding	3" (8 cm.) Comfort
Upholstery	Terra Touch™
Table Weight*	90 lbs. (41 kg.)
Warranty	3 year parts, 1 year labor
Safety Listings	CE marked
OPTIONS & ACCESORIES	
QuickLock Face Rest	
Table Extender	
Inset Shelf (adds 25 lbs. (11 kg.))	
Padding - 4" (10 cm.) Comfort	



# INSTALLATION

## TABLE ASSEMBLY

**NOTE: Retighten all bolts after 4 weeks; the wood will naturally experience slight shrinkage, which can cause the bolts to loosen.**

### SUPPLIED PARTS:

#### TOOLS


- 1 - Large Allen Wrench
- 1 - Small Allen Wrench
- 1 - Open End Wrench

#### LEG HARDWARE

- 8 - Reinforcement Plates
- 8 - Belleville Washers
- 4 - Long Large Bolts
- 4 - Lock Nuts

#### LEG BRACE

- 1 - Wood Leg Brace
- 2 - Short Bolts
- 2 - Barrel Nuts

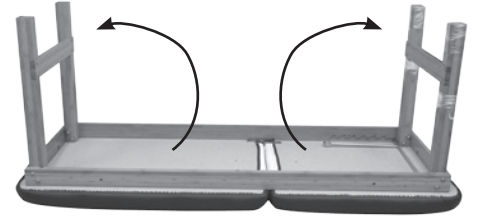



The tables are shipped complete in one box. Follow the step below to fully assemble the table:

1. Remove the table from the box and place it down on the floor with the top down. Remove all hardware and the Leg Brace.
2. Unfold the legs to a vertical position and remove any protective wrappings.

3. Install the Long Large Bolts, Plates, Washers and nuts, as shown, one bolt on each leg. Using the Large Allen Wrench and Open End Wrench, tighten every bolt/nut connection -- 8 total -- as tight as possible.


4. Install the barrel nut into the Leg Brace being sure the screw slit is horizontal. Install the Short Bolt through the leg rail. Start the bolt into the barrel nut using your fingers first. After the bolt has started, tighten the bolt using the small Allen Wrench. Make sure the Leg Brace is flush with the adjoining rails while tightening bolts. Repeat on opposite end of the Leg Brace.

5. Turn your table right side up. Get up on the table and rock back and forth to check that all bolts are tight. The table should be very stiff. If not, check and tighten the bolts as necessary with the supplied tools.

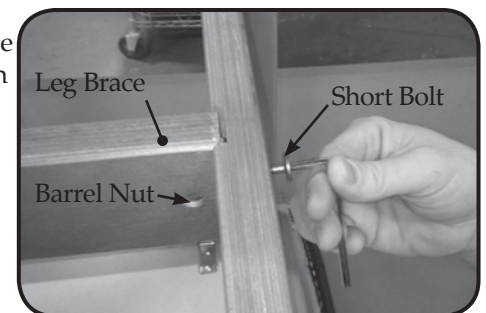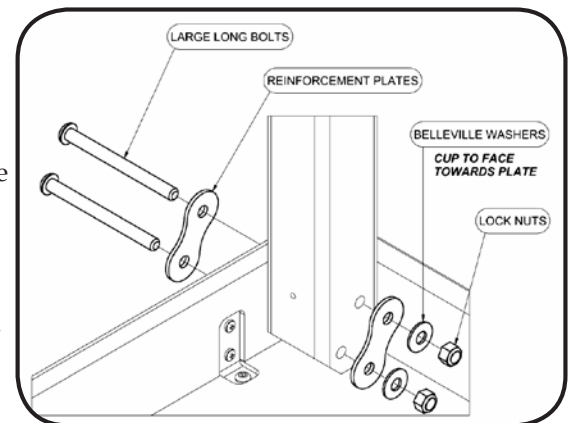


## SHELF ATTACHMENT

If you have ordered a shelf, drop the shelf into place with the finished side up. The shelf has grooves at both ends that enable it to be held firmly in place by the legs. If necessary, the shelf can be further stabilized by installing the enclosed Velcro™ strips. Place strips on each corner of the shelf's bottom side, and on each end of the rails. Make sure not to mismatch Velcro™ strips; place all "hook" or all "loop" pieces on either the rail or shelf.

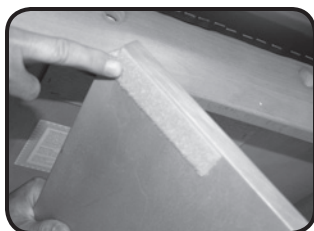


## DIRECTIONS FOR USE


### PAPER ROLL HOLDER USE

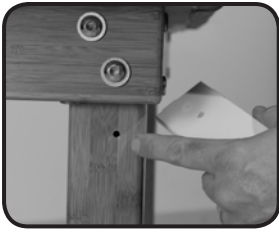

After the table is set-up, locate the hole in the leg.

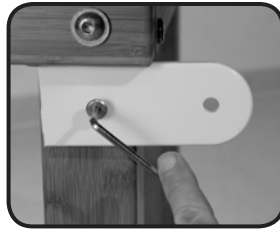

Using the supplied screw & allen wrench, secure the paper roll holder bracket into place. Repeat step for the opposite side.

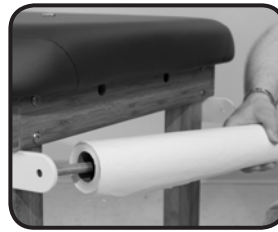

Locate the wooden dowel & feed it thru your paper roll. Place the wooden dowel notch in the hole located in the paper roll holder bracket.



To secure the wooden dowel on the other side, pull out the paper roll holder bracket slightly & insert the notch into the hole.

### WAVE BACKREST USE



#### CAUTION

Do not adjust the Backrest with a patient on the table.



#### TO RAISE THE BACKREST

Lift the backrest to the desired height and release. The Backrest support arm will lock in the nearest slot in the side rail.



#### TO LOWER THE BACKREST

Lift the backrest slightly, lift and hold the support arm out of the slot and lower the backrest to the desired position.


### TABLE EXTENDER




#### CAUTION

Do not put excessive weight or pressure on the table extender.

The Table Extender lengthens the table for your tallest clients and can improve access to your patient.

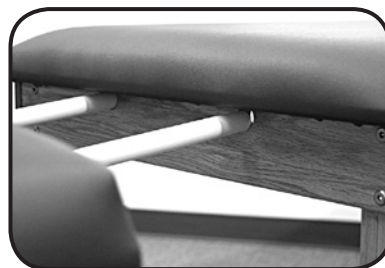

Locate the Accessory attachment outlets.

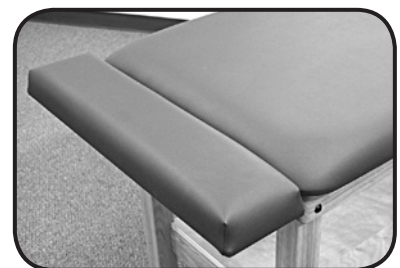

Unfold the dowels and insert into the outlets.

## DIRECTIONS FOR USE

### QUICKLOCK FACE REST


\* U.S. Patent #5,177,823



#### CAUTION

Do not put excessive weight or pressure on the Quicklock Face Rest. Do not extend the dowels of the Quicklock Face Rest 3" (7.6 cm.) from the table top. The Maximum Weight Capacity for the QuickLock Face Rest is 25 lbs. (11 kg.)

Double articulating action provides nearly limitless positioning options with the QuickLock™ Face Rest. The adjustable pad takes pressure off sensitive facial areas while providing superb support. The QuickLock™ Face Rest can be used in both prone and supine positioning and folds out of the way when not in use. Adjustments are made easily by loosening the cam locks, adjusting for height and angle and then locking the cam locks into the desired position.

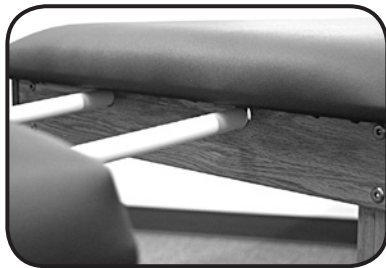

**Important Safety Note:**  
Maximum Distance from edge  
of Table Top= 3" (7.6 cm.)



Insert dowels into  
Accessory attachment out-  
lets.



Loosen Cam Locks &  
adjust face rest into place.



Tighten the Cam Locks.



The Face Rest can be used  
for either prone or supine  
positioning.

## CLEANING & DISINFECTION / INSPECTIONS / WARRANTY

---

### CLEANING & DISINFECTION

Use a 10% sodium hypochlorite (bleach) solution or *Recommended Disinfectants* on all surfaces. Clean all sides of each upholstered section. Follow the directions on the disinfectant and wipe off excess.

#### *Recommended Disinfectants*

Protex, MadaCide, Accell TB, Virox®

**Note: Damage caused by unapproved substances will not be covered under the warranty.**

*DO NOT* use citrus based cleaners or other strong cleaners, such as alcohol, acetone, higher concentrations of bleach or other products that contain high concentrations of these substances.

*DO NOT* expose the fabric to temperatures below 50°F/10°C or above 104°F/40°C.

*DO NOT* expose the fabric to direct sunlight, adhesives, liquids, or abrasive materials.

### INSPECTIONS & MAINTENANCE

RECOMMENDED REGULAR INSPECTIONS (monthly or local standard)

- Visually inspect components for obvious damage that could cause problems during operation.

RECOMMENDED PERIODIC INSPECTIONS (yearly or local standard)

- Visually inspect components for obvious damage that could cause problems during operation.
- Check for abnormal noises.
- Replace any missing or illegible labels.
- Check that all fasteners are present and fastened securely.
- Clean unusual buildup of dirt on the table and/or parts of the table not normally cleaned on a regular basis.
- Check for tears or cracks in the upholstery.

MAINTENANCE

- Retighten all bolts after 4 weeks; the wood will naturally experience slight shrinkage, which can cause the bolts to loosen.
- Oakworks tables/chairs are designed and built to provide many years of dependable service. Please follow local regulations and laws when disposing of the components used in this table.

### WARRANTY

**View complete warranty details at [www.oakworks.com](http://www.oakworks.com)**

# MODEL NUMBER & SERIAL NUMBER

## MODEL & SERIAL NUMBERS



The model number and serial number are located under the top of the table.

Model Number


Serial Number



Model Numbers and Serial Numbers always start with a letter.

## MODEL NUMBER DESCRIPTION

PWMPBRXXXXXXFRTT XX XX

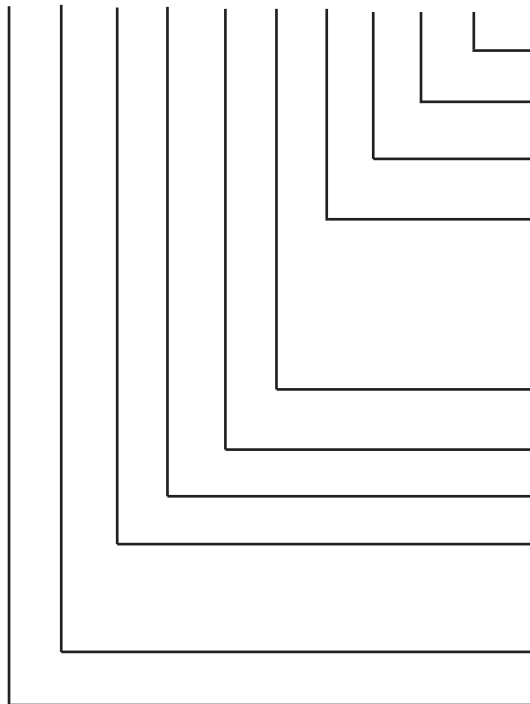

Options

Not Used

Fabric: Terra Touch

Padding

FR=1.75" (4 cm.) Firm Response

SF=2.5" (6 cm.) Semi-Firm

PL=3" (8 cm.) Comfort

AC=4" (10 cm.) Comfort

Table Top Length

Minimum Table Height

Table Width

Top

FT=Flat Top

BR=Wave Backrest

Base

Table

PW=Powerline Series

SE=Seychelle Series

# SPECIFICATIONS

---

PRODUCT SPECIFICATIONS	
Weight	90-115 lbs. (41-52) kg.
Shipping Weight	105-130 lbs. (48-59 kg.)
Weight Limit	500 lbs. (226 kg.)

ENVIRONMENTAL CONDITIONS			
Conditions	Temperature	Humidity	Atmospheric Pressure
Normal Use	50° (10°C) to 104° (40°C)	20% to 60% RH	98 to 105 kPa
Storage & Transport	-20° (-29°C) to 135° (57°C)	20% to 95% RH	98 to 105 kPa

---

**THIS PAGE IS INTENTIONALLY LEFT BLANK**

# USER MANUAL

---

## OAKWORKS® Treatment Tables

### CONTACT INFORMATION:

#### OAKWORKS® Inc.

923 East Wellspring Road  
New Freedom, PA 17349

Phone: 717-235-6807

FAX: 717-235-6798

[www.oakworks.com](http://www.oakworks.com)

---

**EC**

**REP**

#### EMERGO EUROPE

Molenstraat 15  
2513 BH, The Hague  
The Netherlands

For user manuals & instructional videos  
related to your product, please visit  
<http://www.oakworks.com/manuals.asp>

Manual Part Number MMMNST0005-EN

Revision: E

Revision date: 04/24/2015

English, Printed in USA



# OAKWORKS®