OXYMIZER Device

DIRECTIONS

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
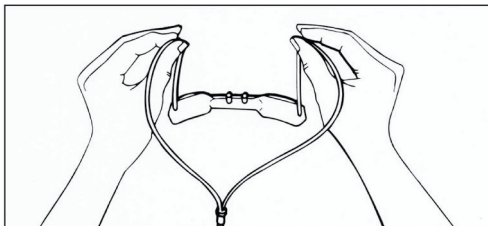
- How often and how long you receive oxygen from the system.
- The flow rate to use. (Note: The flow rate indicated for use with the OXYMIZER device may be 1/4 - 1/2 of the amount prescribed with a non-oxygen-conserving cannula.)
- When to replace your OXYMIZER device.

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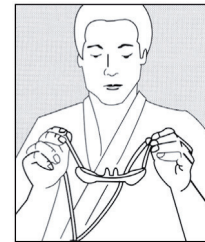
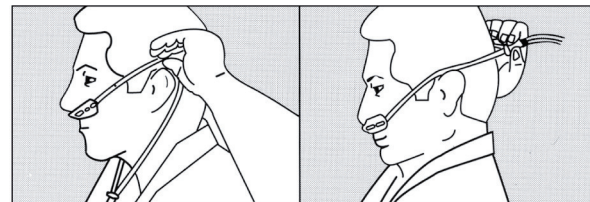
3. Place the flexible plastic tubing over your ears (as shown in diagram below) and under your chin or pass the tubing over your ears and around the back of your head.

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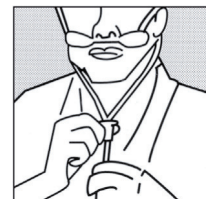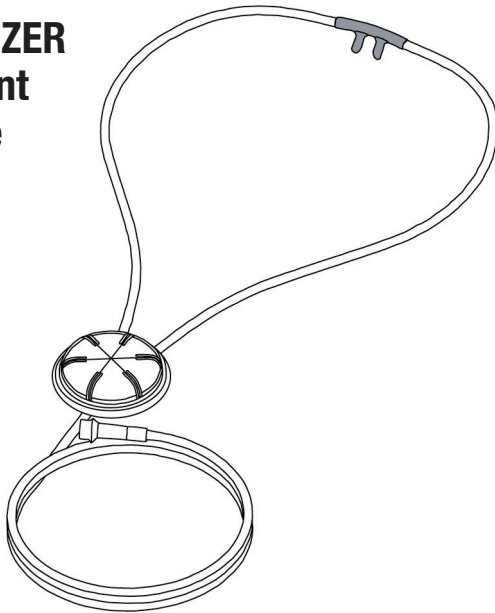
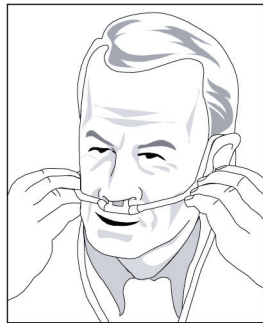
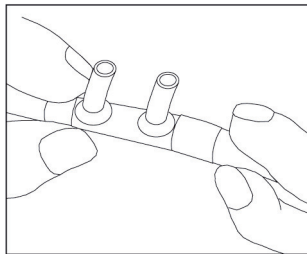
HUMIDIFICATION UNNECESSARY

Since less total oxygen is passing over the nasal mucosa, the drying effect of oxygen is significantly less and supplementary humidification can usually be eliminated. Since excessive moisture could impede the membrane action of the OXYMIZER device, use with humidifiers should be avoided.

OXYMIZER®
DISPOSABLE OXYGEN-CONSERVING DEVICES

**CODE #0-224
FOR USE IN OXYGEN THERAPY**

**DISPOSABLE-
FOR SINGLE PATIENT USE ONLY
DIRECTIONS FOR USE ENCLOSED**

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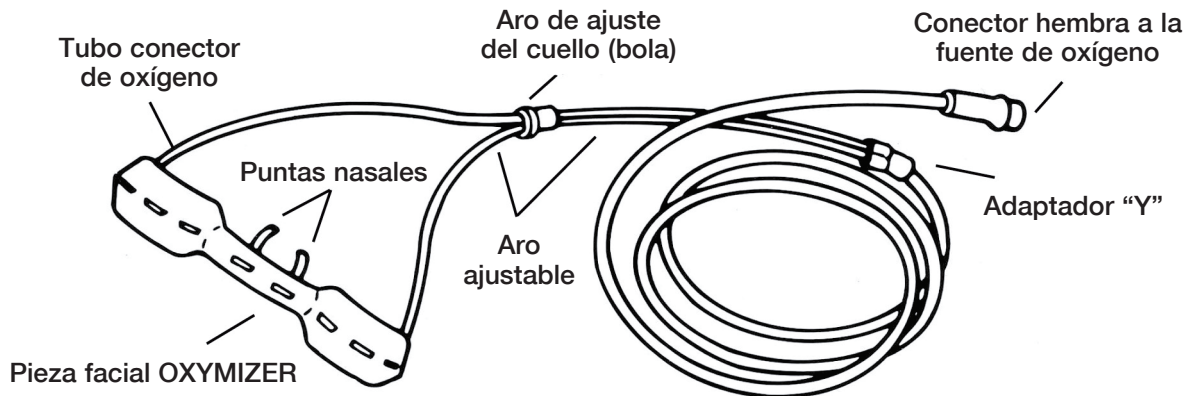
(01)00814470020129

LOT

© 2008 CHAD Therapeutics
Made in China
PM-00052/16/D

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R_x ONLY

drive™

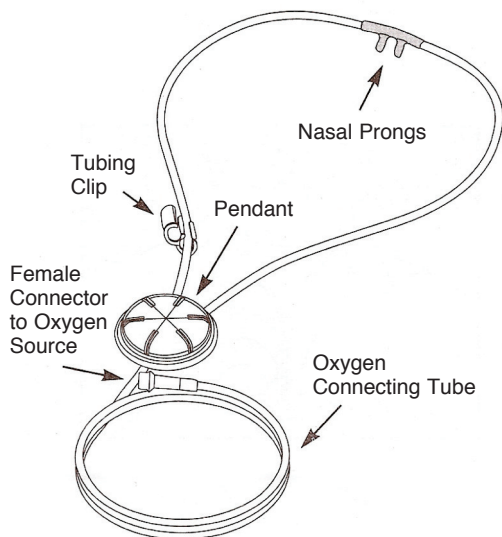
Inovo, Inc
401 Leonard Blvd. North
Lehigh Acres, FL 33971
Toll-free: 877-224-0946
www.drivemedical.com

(01)00814470020129

LOT

© 2008 CHAD Therapeutics
Fabriqué en Chine
PM-00052/16/D

USER DIRECTIONS:

Your doctor or healthcare professional has prescribed the OXYMIZER oxygen-conserving device for you. Used properly, this device will provide you with the oxygen you need, as well as these added benefits:

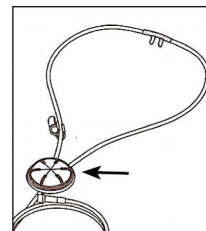
- It will take a lower flow rate to get the same amount of oxygen into your body, which can be as much as 75% less (depending on the flow rate) than a standard nasal cannula.
- If you use a portable oxygen system, it will last longer and allow you to be away from your main oxygen source for a longer period of time; or, you may be able to use a smaller, lighter weight portable system.

It is important that you follow your physician's instructions carefully concerning:


- How often and how long you receive oxygen from the system.
- The flow rate to use.
(*Note: The flow rate indicated for use with the OXYMIZER device may be 1/4 - 1/2 of the amount prescribed with a non-oxygen-conserving cannula.*)
- When to replace your OXYMIZER device.

HOW TO PUT ON THE OXYMIZER PENDANT


1. Uncoil the tubing completely. Hold the device so that the front of the Pendant is facing away from you. (*Note: The front of the Pendant is designated with slots.*)

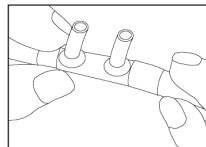2. Position the nasal prongs so that they fit comfortably into your nostrils. The prongs should extend well into and point toward the back of your nose.

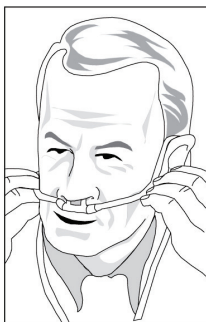
3. Position the nasal prongs and extend the tubing across your cheekbones.

4. Place the tubing around your ears and adjust each side so that you have a snug, comfortable fit. Do not allow the tubing to go completely around your Neck. This may create a choking hazard.

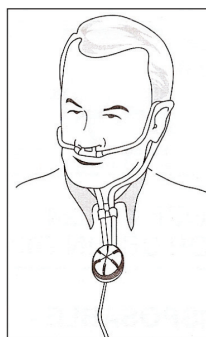5. Attach the tubing clip to join both sides of the cannula tubing. Slide the clip upward beneath your chin to hold the tubing in place (you may choose not to use the clip simply by removing it).

6. The Pendant portion should rest against your chest below your throat. You may want to tuck it under your shirt or blouse.

7. Connect the female connector to your oxygen source.

8. Turn on the oxygen and set the flow rate as prescribed by your doctor or health care professional. (*NOTE: The flow rate prescribed for use with this device may be 1/4-1/2 that prescribed with a non-oxygen-conserving cannula.*)

REPLACEMENT:

The OXYMIZER Pendant device should be replaced after approximately three weeks. More frequent replacement may be needed for cleanliness and sanitation, depending on conditions of use.

TODAY'S DATE:	REPLACE ON:

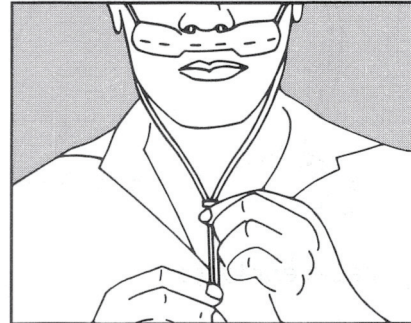
HUMIDIFICATION UNNECESSARY

Since less total oxygen is passing over the nasal mucosa, the drying effect of oxygen is significantly less and supplementary humidification can usually be eliminated. Since excessive moisture could impede the membrane action of the OXYMIZER Pendant device, use with humidifiers should be avoided.

CLEANING

The disposal OXYMIZER oxygen-conserving devices are clean and ready to use, and are supplied in individual containers. The outer surface may be cleaned with soap and water, but care must be taken not to allow the membrane within the reservoir to become wet.

You may also be interested in the original OXYMIZER oxygen-conserving device with the oxygen-saving mechanism contained in a facepiece, which is ideal for use at home (see illustration below).

DISPOSAL

Disposal for your device should be done in accordance with local regulations.

SPECIFICATIONS

Max. Inlet Pressure	50 psi
Max. Flow (LPM)	15 LPM
Operational Temp.	0°C to 50°C (32°F to 122°F)
Storage Temp.	-40°C (-40°F) to 60°C (145°F)

OXYMIZER® PENDANT
DISPOSABLE OXYGEN-CONSERVING DEVICES

CODE #P-224
FOR USE IN OXYGEN THERAPY

**DISPOSABLE-
FOR SINGLE PATIENT USE ONLY**
DIRECTIONS FOR USE ENCLOSED

CONTENTS: One (1) OXYMIZER Pendant disposable oxygen-conserving device with curved tubular nasal prongs, six (6) feet of connecting tube, and a female connector.

R_x ONLY

MANUFACTURED FOR:

drive™

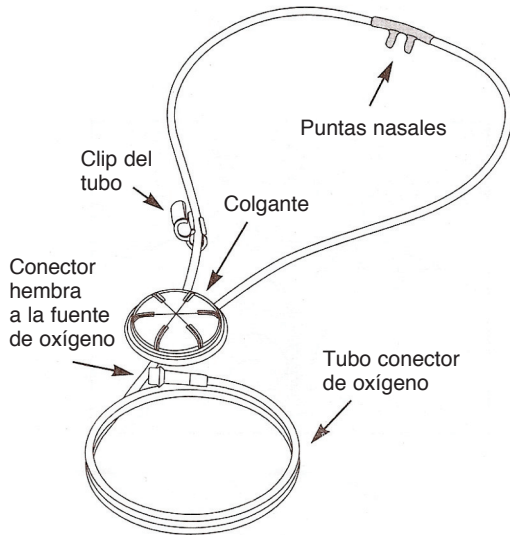
Inovo, Inc
401 Leonard Blvd. North
Lehigh Acres, FL 33971
Toll-free: 877-224-0946
www.drivemedical.com

(01)00814470020112

LOT

© 2008 CHAD Therapeutics
Made in China
PM-00053/16/D

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
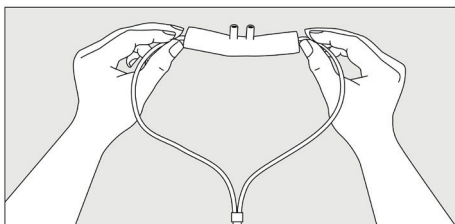
- How often and how long you receive oxygen from the system.
- The flow rate to use.
(Note: The flow rate indicated for use with the OXYMIZER device may be 1/4 - 1/2 of the amount prescribed with a non-oxygen-conserving cannula.)
- When to replace your OXYMIZER device.

HOW TO WEAR THE OXYMIZER DEVICE



1. Slide the loop adjustment collar (bola) down to enlarge the size of the loop.

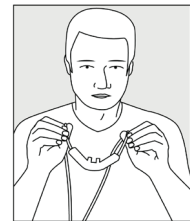
2. With the back of the facepiece toward you, pick up the OXYMIZER with one of the small flexible plastic tubes in each hand, as if putting on a pair of eyeglasses.

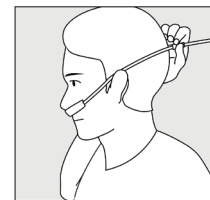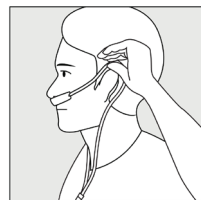
3. Place the flexible plastic tubing over your ears (as shown in diagram below) and under your chin or pass the tubing over your ears and around the back of your head. Do not allow the tubing to go completely around your Neck. This may create a choking hazard.

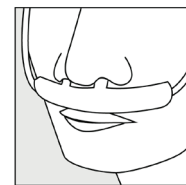4. The facepiece should rest on your upper lip under your nose with the oxygen-delivering prongs extending well into and pointing towards the back of your nose.

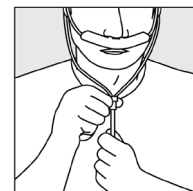
5. Slide the bola up toward your chin so that the cannula fits snugly and comfortably against your face.

6. Connect the female connector to your oxygen source.


7. Turn on the oxygen and set the flow rate as prescribed by your doctor or healthcare professional. (Note: The flow rate prescribed for use with this device may be 1/4 - 1/2 of the amount prescribed with a non-oxygen-conserving cannula.)

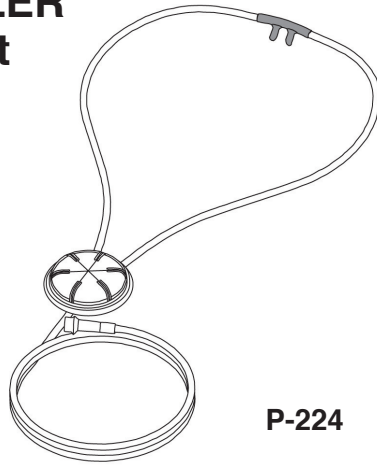**OXYMIZER
Pendant
Device**

P-224

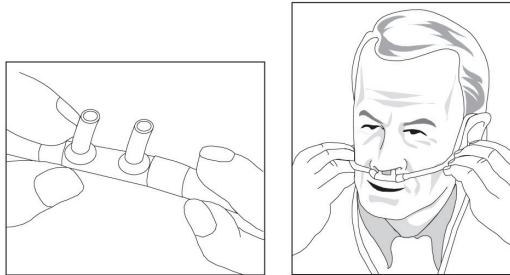You may also be interested in the OXYMIZER Pendant device (pictured above and below). In this version, the oxygen-conserving mechanism is contained within the pendant, which rests against your chest and can be easily concealed by your clothing.

SPECIFICATIONS

Max. Inlet Pressure	50 psi
Max. Flow (LPM)	20 LPM
Operational Temp.	0°C to 50°C (32°F to 122°F)
Storage Temp.	-40°C (-40°F) to 60°C (145°F)

REPLACEMENT

The OXYMIZER oxygen-conserving devices should be replaced after approximately three weeks. More frequent replacement may be needed for cleanliness and sanitation, depending on conditions of use.

TODAY'S DATE:	REPLACE ON:

HUMIDIFICATION UNNECESSARY

Since less total oxygen is passing over the nasal mucosa, the drying effect of oxygen is significantly less and supplementary humidification can usually be eliminated.

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CONTENTS: One (1) OXYMIZER disposable oxygen-conserving device with curved tubular nasal prongs, 6 feet of connection tubing, and one (1) female connector.

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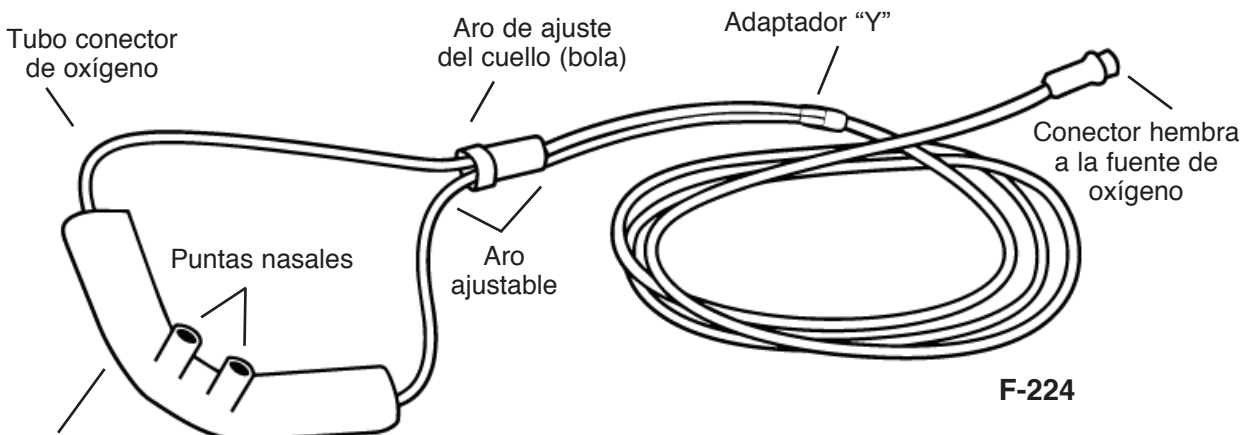
(01) 00814470020259

© 2008 CHAD Therapeutics
Made in China
PM-00057/16/C

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