

# Changing Table

4521-Large (74.8" long)

4521-01-Small (59" long)

9245-21- Side Rail

## PRECAUTIONS

Professional installation is recommended.



Small Changing Table



Small Changing Table with Rail



## INSTRUCTIONS FOR USE

1. Lay stretcher on its side and extend the legs.

2. Stand stretcher up, pull out wall brackets, and extend legs outward to lock in place.



4. To fold table up, grab bar and pull towards your body. This will stretch the spring and disengage the leg from the socket.

3. Install wall brackets. It is recommended that wall brackets be installed by a qualified contractor. (Failure to install the stretcher properly to the wall could result in injury). After wall brackets are mounted properly, remove blocks from the back legs.

(Hardware is not included due to different construction of facility walls. This is one of the reasons why installation should be done by a professional).

NOTE: The back legs are intentionally designed to be 1cm shorter than the front legs and will be off the floor while the table is in use. This design allows for ease of folding and storing and will not impact patient safety.



5. Install wall bracket to hold the table in place when folded.



### Limited Product Warranty

Patterson Medical warrants that all products shall be free from manufacturing defects for a period of one year from the date of shipment. OTHER THAN THE FOREGOING WARRANTY, PATTERSON MEDICAL MAKES NO REPRESENTATION OR WARRANTY OF ANY KIND, EXPRESS OR IMPLIED, AS TO MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE OR ANY OTHER MATTER RELATING TO THE PRODUCT. Customer must notify Patterson Medical of any manufacturing defect within five days of discovery of the defect. Upon a determination by Patterson Medical that the product is defective, Patterson Medical sole and exclusive obligation shall be to provide customer with a replacement product or a full refund of the purchase price, at the sole discretion of Patterson Medical, and to reimburse customer for any return freight costs. In no event shall Patterson Medical be liable for any other costs or expenses including lost profits or direct, indirect, consequential, incidental or punitive damages. The warranty set forth herein shall govern all purchases, notwithstanding any inconsistent terms and conditions of customer's purchase order.

### WARNINGS

For safety, adhere to all warnings and follow instructions.

Use this product for one patient at a time.

**NEVER** leave patient unattended during use.

**DO NOT** use if parts are missing, broken, damaged or worn.

Stop use of the product until repairs are made using factory authorized parts.

Maximum weight capacity: 250 lbs.

Make sure table is steady on floor and properly attached to wall before use.

Professional installation is recommended.

**Failure to follow these warnings could result in serious injury.**

### CARE

Table can be cleaned regularly with normal household detergent or antibacterial wipes.

Do not use aggressive cleaners.



#### Distributed by: Patterson Medical

28100 Torch Parkway, Ste. 700  
Warrenville, IL 60555  
Phone: 800-323-5547  
Direct Phone: 630-393-6000  
Fax: 800-547-4333  
Direct Fax: 855-480-1946

#### Manufactured for: Patterson Medical

W68 N158 Evergreen Blvd.  
Cedarburg, WI 53012  
Phone: 800-323-5547  
Fax: 800-547-4333

#### Authorized European Representative Patterson Medical, Ltd.

Nunn Brook Road  
Huthwaite, Sutton-in-Ashfield  
Nottinghamshire, NG17 2HU, UK

#### United Kingdom

Phone: 08444 124 330  
Fax: 08448 730 100  
International  
Phone: +44 (0) 1623 448 706  
Fax: +44 (0) 1623 448 784