




# STANDARD MALE URINAL KIT

## URINE COLLECTION SYSTEM

USE AND CARE INSTRUCTIONS  
FOR PRODUCT #4401, #4400 AND #4399

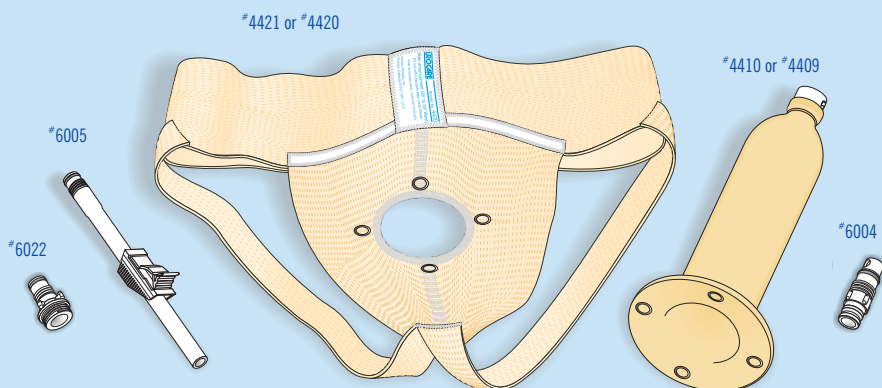

Examine this product carefully. If not entirely satisfied, do not accept.

**UNDER NO CONDITIONS WILL IT BE SUBJECT TO RETURN!**

Sanitary reasons make this ruling absolute.

### KIT CONTENTS:

- 1 Standard Suspensory Garment (#4421 or #4420)
  - 1 Male Urinal Sheath (#4410 or #4409)
  - 1 Standard Bottom Drain Valve (#6004)
  - 1 Straight-thru Adaptor with Clamp (#6005)
  - 1 Male Urinal to Latex Leg Bag Adaptor (#6022)
- Illustrated Use and Care Instructions

## IMPORTANT! READ THIS FIRST:

For best results, certain precautions should be taken before each application. Among them, be certain that: all snaps are firmly fastened to both the garment and the urinal sheath before each use, there are no loose stitches and there are no sharp edges on any of the components.

**CAUTION:** Those with known allergic reactions to synthetic materials, especially latex rubber, should not wear this system. Discontinue use immediately at first signs of any skin or other complications and consult a fitter or physician. If discomfort occurs or prolonged wear affects the performance of the garment, discontinue use. This product is intended for use in medical rehabilitation for urine collection, not as any form of protection during sports.

## DIRECTIONS FOR USE AS A DRIP URINAL

### STEP 1: Trimming the Inner Sheath

Pull out the funnel-shaped, inner sheath as shown in Figure 1. The opening of the sheath should be trimmed to fit snugly around the penis to prevent leakage. If the opening of the sheath is too small or tight, trim off not more than an  $\frac{1}{8}$ " (3mm) at a time. Repeat if necessary until satisfied that you have a comfortable, snug fit. Cut the sheath with a pair of large, sharp scissors to prevent jagged or nicked edges before application.

**CAUTION:** For comfort, the inner sheath is made of thinner, lightweight latex and may tear if cut or trimmed improperly. Be certain there are no signs of jagged or nicked edges before application.

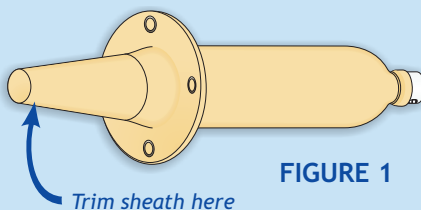


FIGURE 1

### STEP 2: Application of the Urinal Sheath

After the sheath has been trimmed, allow the penis to enter as far as possible so that the end of the inner sheath is as far as possible behind the head of the penis. Return the inner sheath with the penis inserted to its original position inside the out sheath. If the Standard Bottom Drain Valve (#6004) is not already in place, inset into the valve housing at the tip of the urinal sheath and while turning gently clockwise, screw it into position as shown in Figure 2.

**CAUTION:** Do not over-tighten drain valve—doing so may damage the valve housing. Remember to close the drain valve before use. To open the valve for drainage, twist it counter-clockwise  $1\frac{1}{2}$  turns.

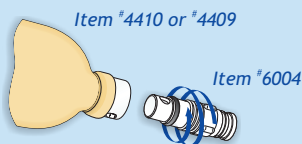


FIGURE 2  
Enlarged view of the sheath's valve housing

### STEP 3: Application and Adjustment of the Suspensory

With the urinal sheath already applied to the penis, pull the garment over the legs passing the tip of the urinal sheath through the opening in the front of the garment. Then snap the urinal sheath firmly into position as shown in Figure 3. Adjustment of the suspensory garment should be made while wearing. Please note: The garment should fit snug around the waist, but not too tight.

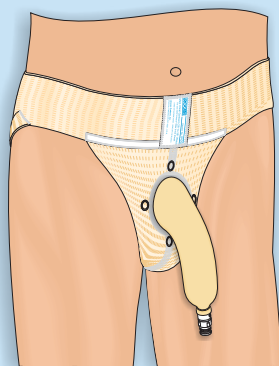



FIGURE 3

Suspensory garment and sheath fitted correctly

## DIRECTIONS FOR USE WITH A UROCARE® LATEX LEG BAG


### STEP 1: Deciding on a Position for the Latex Leg Bag

Follow steps one(1) through three(3) on the previous page. Once the suspensory and sheath are in place it's time to decide on a position for your leg bag. If your preferred leg bag position is either over or below the knee, the use of an extension tube (#6005, supplied, or #6016, not supplied) may be necessary—see Figure 4.

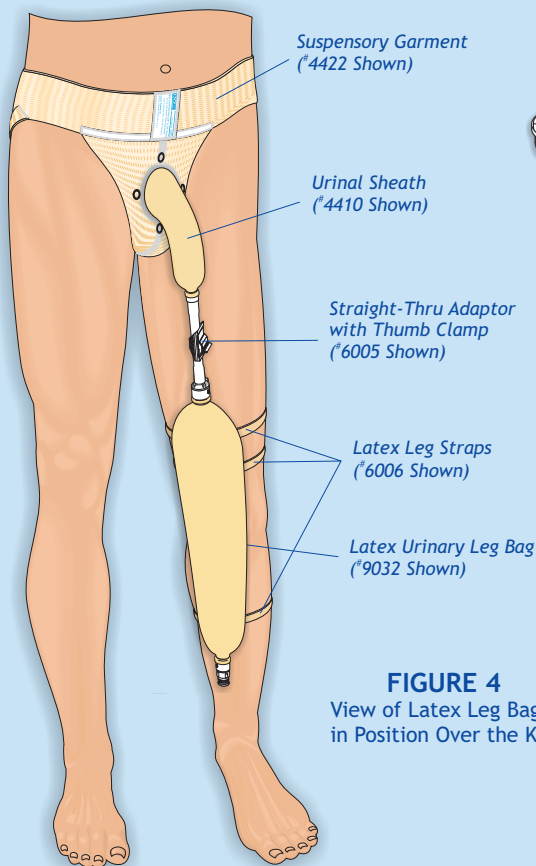

**FIGURE 4**

View of Latex Leg Bag  
in Position Over the Knee

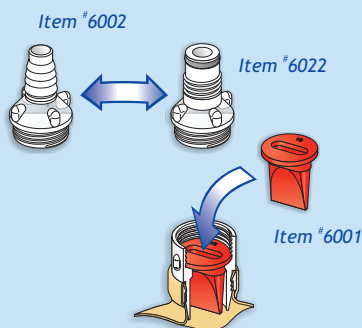

**FIGURE 5**

Enlarged View of  
Latex Leg Bag Top

**IMPORTANT:** To avoid drainage problems, before using this product, read all instructions and information carefully. Understand how it works and how to maintain it properly.

#6000



Black Gasket-Ring used  
to replace the anti-reflux  
valve when not in use.

### STEP 2: Removing the Bottom Drain Valve and Standard Top Adaptor

If you prefer to connect the bag directly to the sheath, both the Standard Bottom Drain Valve (#6004) and the Standard Top Adaptor (#6002) must be removed. Start by removing the drain valve from the sheath by turning it counter-clockwise until it separates from the valve housing, then remove the Standard Top Adaptor from your latex leg bag in the same manor and set them aside—be certain the anti-reflux valve does not fall out or get lost.

### STEP 3: Attaching the Male Urinal Adaptor

With the anti-reflux valve still in place, attach the Male Urinal Adaptor (#6022) to the Top Valve Housing of the leg bag by turning it clockwise until firmly seated in position—see Figure 5.

## DIRECTIONS FOR USE WITH A UROCARE® LATEX LEG BAG

### STEP 1: Deciding on a Position for the Latex Leg Bag

Follow steps one(1) through three(3) on the previous page. Once the suspensory and sheath are in place it's time to decide on a position for your leg bag. If your preferred leg bag position is either over or below the knee, the use of an extension tube (#6005, supplied, or #6016, not supplied) may be necessary—see Figure 4.

### STEP 2: Removing the Bottom Drain Valve and Standard Top Adaptor


If you prefer to connect the bag directly to the sheath, both the Standard Bottom Drain Valve (#6004) and the Standard Top Adaptor (#6002) must be removed. Start by removing the drain valve from the sheath by turning it counter-clockwise until it separates from the valve housing, then remove the Standard Top Adaptor from your latex leg bag in the same manor and set them aside—be certain the anti-reflux valve does not fall out or get lost.

### STEP 3: Attaching the Male Urinal Adaptor

With the anti-reflux valve still in place, attach the Male Urinal Adaptor (#6022) to the Top Valve Housing of the leg bag by turning it clockwise until firmly seated in position—see Figure 5.

**FIGURE 6**

Enlarged View of the  
Urinal Sheath Valve Housing

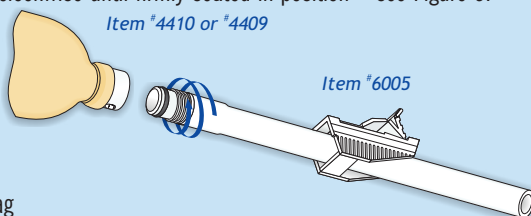

### STEP 4: Attach Straps to the Leg Bag

Whether you are using the latex leg straps supplied or a set of fabric straps, you should attach them to your latex leg bag prior to connecting the bag to the urinal sheath. Feed the strap ends/ties through the strap loops on the latex leg bag. Once the above is completed, the Male Urinal Adaptor already attached to the bag can be connected to the urinal sheath by inserting the tip of the adaptor into the valve housing of the sheath and turning both the adaptor and the bag clockwise until firmly in place—see Figure 4. Once the urinal adaptor has been properly attached to the sheath it's time to secure the leg straps to the leg. This is the final step. At this point you may wish to make some minor adjustments to the suspensory garment, sheath or leg bag position for comfort.

## DIRECTIONS FOR USE WITH OTHER LEG BAGS

### STEP 1: Deciding on a Position for the Latex Leg Bag

Follow steps one(1) through three(3) on the previous page. Once the suspensory and sheath are in place it's time to decide on a position for your leg bag. If your preferred leg bag position is either over or below the knee, an extension tube (No.6005, supplied, or No. 6016, not be supplied) may be necessary.

### STEP 2: Replacement of the Standard Bottom Drain Valve

If you prefer to connect your leg bag directly to the sheath, the Standard Bottom Drain Valve (#6004) must be replaced with the Straight-Thru Adaptor (#6005). Remove the drain valve from the sheath by turning it counter-clockwise until it separates from the valve housing. Then, insert the Straight-Thru Adaptor and turn clockwise until firmly seated into position. CAUTION: Do not over-tighten the Adaptor—doing so may damage the valve housing.

### STEP 3: Attaching the Leg Bag


# IT'S ALL IN THE BAG!

**New high-flow, anti-reflux valve** is self-cleaning and prevents back-flow while allowing **fast**, uninterrupted drainage for **problem-free use**. Don't settle for reflux or back-pressure caused by inferior flutter valves. Nothing beats Urocare's new **high-flow** Little Red Valve for performance, reliability and safety.

**New built-in odor barrier:** We don't give odor a chance! We added a special new odor control barrier to the **inside & outside** of our latex leg bags for double odor protection. This special odor barrier makes Urocare's leg bags virtually **ODOR-FREE**, not just odor-resistant; thereby preventing embarrassing odors **before** they start.

Choose **complete confidence** and **complete protection**—choose Urocare!

*Why settle for anything less?*

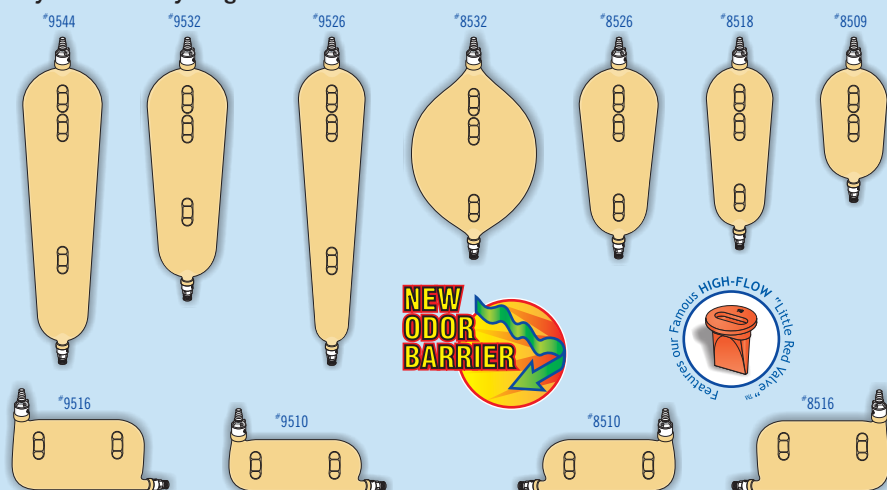

ITEM #	DESCRIPTION	DIMENSIONS†		CAPACITIES†	
		Inches(″)	Centimeters(cm)	Ounces(oz.)	Milliliters(ml.)
9544	Extra-Large	20 $\frac{5}{8}$ x 4 $\frac{3}{4}$	52.4 x 12.1	44	1,300
9532	Large	16 $\frac{3}{4}$ x 4 $\frac{3}{4}$	42.5 x 12.1	32	1,000
9526	Long-Slim	20 $\frac{5}{8}$ x 3 $\frac{3}{4}$	52.4 x 9.5	26	800
9516†	Sport-Right, Large	5 $\frac{3}{4}$ x 10	14.6 x 25.0	16	500
9510†	Sport-Right, Small	4 $\frac{3}{4}$ x 10	12.1 x 25.0	10	300
8532	Large-Oval	13 x 7	33.0 x 17.8	32	1,000
8526	Short-Wide	13 $\frac{3}{8}$ x 4 $\frac{3}{4}$	34.0 x 12.1	26	800
8518	Medium	13 x 3 $\frac{3}{4}$	33.0 x 9.5	18	550
8516†	Sport-Left, Large	5 $\frac{3}{4}$ x 10	14.6 x 25.0	16	500
8510†	Sport-Left, Small	4 $\frac{3}{4}$ x 10	12.1 x 25.0	10	300
8509	Small	9 $\frac{3}{4}$ x 3 $\frac{3}{8}$	24.8 x 9.2	9	275

† Approximate dimensions include adaptors and drain valves. Approximate volumes in fluid ounces and milliliters.

‡ These leg bags are designed to be worn horizontally above the hemlines of skirts, dresses and swimsuits.



## RELATED PRODUCTS

Urocare® Reusable Latex Leg Bags  
Uro-Safe® Disposable Vinyl Leg Bags  
Urinary Drainage Systems & Kits  
Uro-Con® & Uro-Cath® Male External Catheters  
Drain Valves & Adaptors  
Drainage Bottles  
Uro-Prep® Protective Skin Barrier Wipes  
Fabric Leg Bag Holders & Straps  
Catheter Straps  
Uro-Bond® Brush-on Silicone Adhesive  
Urofoam® Adhesive Foam Strips  
Adhesive Remover Pads  
Urolux® Urinary Appliance Cleanser & Deodorant  
Connectors & Clamps  
Extension & Drainage Tubing  
DRI Excel™ & DRI Eclipse™ Bed-Wetting Treatment Alarms

For more information please contact us at **1-800-423-4441** or visit us on the internet at: [www.urocare.com](http://www.urocare.com). To place an order, please contact your health care supplier.

Manufactured and distributed by:

Urocare Products, Inc.  
2735 Melbourne Avenue  
Pomona, CA 91767-1931  
U.S.A.

Tel: (909) 621-6013 • Fax: (909) 621-4436  
Internet: [www.urocare.com](http://www.urocare.com)



PLEASE  
RECYCLE

MADE IN  
U.S.A.

FORM NO: PI4400-89019000C