

ALTERNATING PRESSURE PAD

QUICK START GUIDE

WHAT'S INCLUDED

- 1x Pressure Pad
- 1x Inflation Tubes
- 1x Alternating Pressure Pump

OVERVIEW

We are constantly answering questions and recording helpful videos to make using your Vive Alternating Pressure Pad as easy as possible. Check out the included links and QR codes to help you through the process.



To see all FAQs in one place, visit vhealth.link/sqp

QUICK SETUP



Want to watch us do this? Check out vhealth.link/x10 for an easy instructional video.

1. Unroll the Pressure Pad and lay it flat on your mattress. Make sure the inflation tubes are facing up and positioned at the head or foot of the bed, depending on where you plan to place the Pressure Pump.
2. Use Built in metal hooks to Hang the Pressure Pump over the bed frame at the foot of the bed (Recommended). You may also lay the pump flat on a table or floor.
3. Connect the inflation tubes to the Pressure Pump and Pressure Pad.
4. Plug the Pressure Pump into electrical outlet. Power supply is compatible with USA Standards (110v).
5. Turn the Pump on and adjust the pressure dial to “Firm” setting to speed up inflation process. Wait until the mattress has been fully inflated before using (takes approx. 10 minutes to fully inflate).

6. Set the variable pressure dial to ideal setting for the user depending on weight and comfort. We recommend starting at “Firm” and adjusting based on the user.

HOW IT WORKS



Want to watch us do this? Check out vhealth.link/vi2 for an easy instructional video.

The Vive Alternating Pressure Pad has 130 bubble air cells. The air cells are divided into two different sections. Each section is connected separately to the pressure pump. The pressure pump automatically alternates between sections at 6 minutes intervals.

The pressure dial on the front of the pressure pump allows you to adjust the pad pressure from firm to soft. Regardless of the pressure setting the pump will always alternate in 6 minute intervals.

CLEANING INSTRUCTIONS

Pressure Pad/Pressure Hose

Use cloth with a mild detergent solution or a surface detergent/ disinfectant for cleaning the pad and hose.

REPAIR GUIDE



Want to watch us do this?
Check out [vhealth.link/mjd](https://vivehealth.link/mjd)
for an easy instructional video.

SMALL PUNCTURES: Use the patch kit.

1. Fully deflate the unit.
2. Take the patch and add glue around the edges.
3. Firmly place the patch over the puncture.
Hold for 20 seconds until glue is dried.
4. Let stand for 30 minutes.

SPECIFICATIONS

Pump

- Interval Duration: 6 Mins.
- Air Output: 6-7 L/Min.
- Pressure Range: 35~130 mmHg
- Electrical Power Supply: 110-120 Volts

Mattress

- Dimensions: 36" x 78" x 3" (inflated)
- Materials: PVC (EN-71) 0.3mm

WARNING

- Weight Capacity: 300 lbs.
- Do not use corrosive cleaning products such as industrial degreasers or acetic solvents.
- Keep away from sharp objects as they may puncture the pressure pad.

vive®

Distributed by

vive
health

8955 Fontana Del Sol Way
Naples, FL 34109
1-800-487-3808
www.vivehealth.com