

## DESCRIPTION

Wiesner incontinence Clamp is an external medical device used to control leakage of urine, the device makes a compression of the urethra, preventing the flow of urine.


Our product is characterized by its ergonomic design, which provides user both safety and comfort.

We are the only ones on the market that uses silicone pads allowing proper cleanliness and durability.


The use of these devices has a huge impact on the quality of life for patients, restoring their confidence and allowing them to enjoy activities that previously could not.

Visit our webpage: [www.wiesnerhealth.com](http://www.wiesnerhealth.com)


## FEATURES

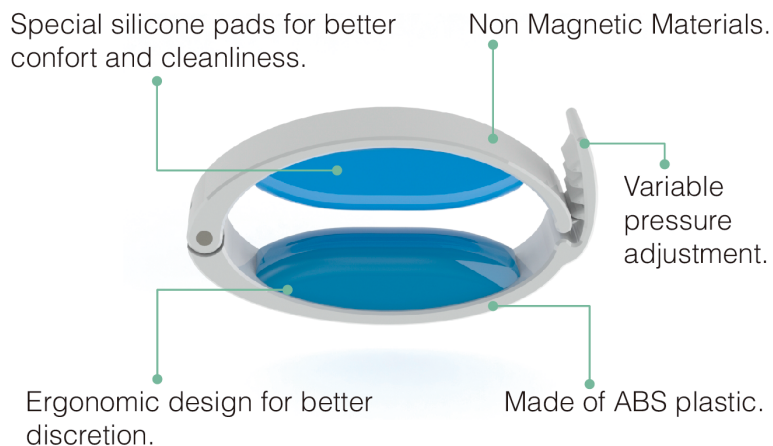


## INSTRUCTIONS OF USE

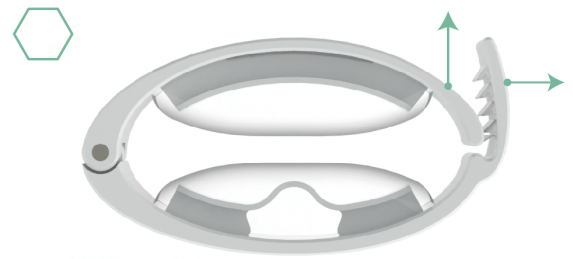

Open the clamp by pulling the retention and lifting the upper part.

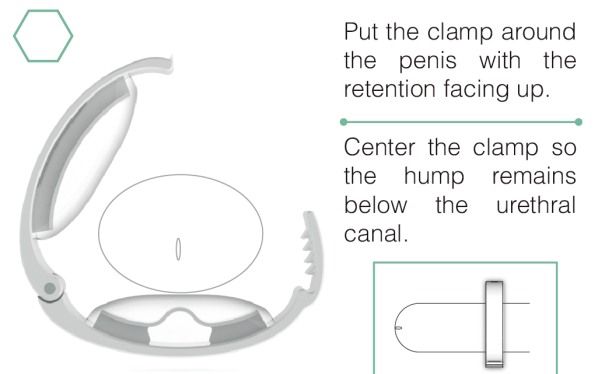

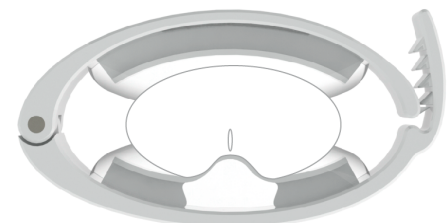


Close the clamp to the level of confort you want, notice that a very light pressure will prevent any leakage.



Do not set the pressure to high so it could interfere with the blood circulation.

## WASHING INSTRUCTIONS

-  Wash with mild soap and water.
-  Dry the clamp with a towel or let it air dry.
-  Do not use bleach or any other detergent.

Wiesner Incontinence Clamp is a non sterile Device.

Caution: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.