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13-2845-0-00000 REV B to Rev C - 2021-02-23

## APPLICATION INSTRUCTIONS

Before applying the brace, please read these instructions completely and carefully. Correct application is vital to the proper functioning of the brace.

### INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

### INDICATIONS:

This CTS Wrist Brace is designed for immobilisation of the wrist while allowing the fingers to move free. It provides wrist support and ideal for Carpal Tunnel Syndrome (CTS), wrist sprains, wrist strains and wrist tendonitis (Tenosynovitis). The support can also be used to help reduce pain while recovering from injuries as well as help to prevent any further injuries.

### PERFORMANCE CHARACTERISTICS:

- Soft-good/ semi-rigid designed to restrict motion through elastic or semi-rigid construction.
- Providing mild compression of limb or body segment through elastic construction.
- Providing immobilization or controlled movement of the limb or body segment.

**CONTRAINDICATIONS:** None.

### APPLICATION INSTRUCTIONS:

- A** Place hand in splint with stay aligned with arm and the thumb resting in the side slot. (Fig. 1)

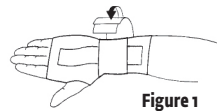
Figure 1

- B** Attach straps in sequence A, B, and C. While applying use constant tension to elastic strap, wrap and attach. (Fig. 2)

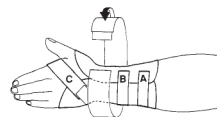
Figure 2

### USE AND CARE:

Hand wash in cold water using mild soap, rinse thoroughly. AIR DRY. Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

### WARNINGS AND PRECAUTIONS:

- Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene by products.
- Do not use over open wounds.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.

**NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

CAUTION: Please be advised that dj Orthopedics, LLC does not recommend simultaneous wearing of functional knee braces bilaterally (right and left legs), during activities (ie... skiing) that may cause the braces to contact each other, resulting in potential loss of control and injury. Functional knee braces are not recommended to be worn over clothing. Consult your licensed Healthcare professional for your activity limitations.

CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PROFESSIONAL

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©2021 DJO, LLC